



SureSmile® Advanced

Staff Level 1 - Module 5

Inserting and Managing SureSmile Wires

Tracking Wire Shipments

Table of Contents

- Best Practices for Wire Management..... 2
- Lesson 1: Working with Custom Wires..... 4
- Lesson 2: Checking the Rx Box..... 12
- Lesson 3: Ordering/Cancelling Wires..... 17
- Lesson 4: Opening Finished Wire Orders..... 22
- Lesson 5: Navigating the Wire Design Workspace 26
- Lesson 6: Finding/Recording IPR 31
- Lesson 7: Getting Ready for Wire Appointments..... 34
- Lesson 8: Following the Wire Insert Protocol..... 37
- Lesson 9: Evaluating Treatment Progress..... 43
- Tracking Wire Shipments 47
- Checking in Wires 50
- Finding Wire Shipments..... 54
- Trouble-shooting Wire Tracking 56
- Learning Resources..... 57

Best Practices for Wire Management

Add procedure codes for SureSmile wire appointments

For example:

ASW1 - insert first wire (you need to find the wire box)

AWS2 - evaluate wire 1 (don't need to look for wire box but this code allots certain time in the schedule for a SureSmile wire check. Order wire mods as needed.)

AWS3 - insert modified wire (you need to find the wire box)

Schedule just the right amount of time

- At the therapeutic scan appointment, update the wire insertion date on the timeline.
- Set the initial wire insertion date approximately 6 weeks after the scan to deliver efficient treatment while allowing enough time for the doctor plan and wire shipment. Remember to allow for holidays and vacations.
- In most practices, the doctor places the wire order. You can send advanced staff to Staff Level 2 training to learn how to assist the doctor with wire order tasks.

Set up a receiving process and storage system

- Delegate the task of receiving wire shipments and filing wires.
- Use a bar code scanner to improve the accuracy and speed of checking in wire shipments.
- Create a storage system for your wires, organized by patient last name. Tip: the boxes fit into CD organizers.
- Group the patient's wire sets with rubber bands to keep them together and make them easy to pull by patient.

Check wires have arrived in time for appointments

- To figure the date the wires must be ordered to arrive on time, count back 11 business days (don't count holidays or weekends).
- One week ahead, run a report of wire appointments. For wires not arrived yet, go to Shipments tracking and determine if wires have shipped. If shipped, look up the ETA and check again on that day. If not, check weekly and then daily for a shipped status.
- If the doctor is late ordering the wires and there is still time to receive them, use flags to mark urgent patients. (Reinforce that the doctor should sort on wire insertion date to prioritize his/her work.)
- Reschedule the patient if you determine the wires will not arrive on time. Or, you can perform a wire check of the current wires and reschedule the patient to return on a shorter rotation.
- If wires are transferred to a satellite office, use the Shipments tracking page or a log to manage transfers.

Manage unused wires and finished patients

- Put unused wires in a basket. At the end of the day, re-shelve them and retain until the end of treatment.
- If the doctor orders a wire modification (i.e., changes the prescription), discard any extra wires based on the old prescription
- When the patient is finished, separate the paper from the plastic prescription box and recycle.

Lesson 1: Working with Custom Wires

Since you have experience installing preformed wires, this module focuses on custom wires. Tips for working with Nickel-titanium (NiTi) wires are also included since this material is especially effective in SureSmile treatment.

Objectives

By the end of this section, you will be able to:

- Recognize advantages of NiTi
- List tips for inserting NiTi
- Recognize SureSmile®'s wire options
- Identify orientation issues for custom wires
- Avoid wire/bracket breakage
- Engage custom wires
- Identify purpose of staged wires

Has everyone seen the TC demonstration of custom wires?

Suggested Script

Traditional Orthodontics

In traditional orthodontics, a series of wires are used to move the teeth through paces of treatment. After initial leveling and alignment, fine-tuning is usually accomplished with stiff wires that must be bent manually. They can apply a great deal of pressure and the doctor places bends at multiple appointments until the final result is achieved.

SureSmile Prescription Archwires

SureSmile archwires work more efficiently than traditional wires for two reasons: they are custom-designed for each patient by the doctor and are made of a Nickel Titanium (NiTi) material developed by NASA called Shape Memory Alloy (SMA). Activated by body temperature, this material provides constant forces to move teeth into the correct position comfortably.

To create a custom prescription, the doctor uses a computer to plan treatment to its conclusion, and a robot to heat the wire and “burn in” custom bends. (In a traditional practice, the doctor cannot place bends in SMA wires since this special material can only be bent at high temperature.) Because SureSmile employs Shape Memory Alloy, the bends are permanent and the wire will “remember” its shape – even after it’s distorted to fit a patient’s crooked teeth. Once it is inserted, the patient’s body heat activates the wires’ memory. It gently exerts energy to return to the shape that reflects the doctor’s plan for treatment.

Recognize Treatment Advantages of NiTi

DISCUSSION: NiTi wires have several properties that make it advantageous for orthodontic treatment:



Because SureSmile wires are produced by robots that can heat the material, SureSmile can take advantage of the shape memory property of NiTi.

Shape memory is achieved by first establishing a shape at temperatures near 482° C. If the appliance wire is then cooled and formed into a second shape and heated through a lower transition temperature range (TTR), the wire will return to its original shape.

Ravichandra Sekhar Kotha, Rama Krishna Alla, Mohammed Shammam, Rama Krishna Ravi, An Overview of Orthodontic Wires: Trends Biomater. Artif. Organs, 28(1), 32-36 (2014)

List Tips for Inserting NiTi

Who has worked with NiTi before? You may already know that this alloy is more elastic below body temperature. That means you can cool the wire to help you insert it more easily. Use a refrigerant spray you normally stock such as Essix Freeze Spray®.

You can still cool the wire once it is placed, and you are attempting to tie it in. Simply press an ice cube against the wire where you want to increase flexibility to gain a few seconds of working time. Or, you can spray pliers with the refrigerant spray and then press against the wire to cool it.



Recognize SureSmile's Wire Options

While NiTi has many uses, several wire materials and cross-sections are supported to accommodate various treatment needs as shown in the following tables.

Indication of Use

Stiffness	Type of Material	Alignment	Leveling	Archwidth	Torque	Elastics		Stage of Treatment
Low	Nickel Titanium (NiTi)*	✓✓✓	✓✓ <small>(w/ auxiliaries ✓✓✓)</small>	✓ <small>(w/ auxiliaries ✓✓✓)</small>	✓ <small>(w/ auxiliaries ✓✓✓)</small>	✓✓	✓	Early to Final
	Beta Titanium (Beta-Ti)**	✓	✓✓✓	✓✓✓	✓✓✓	✓✓✓	✓✓✓	Mid to Final
High	Elgiloy**	✗	✓✓	✓✓✓	✓✓✓	✓✓✓	✓✓✓	Final

- ✓ = Adequate
- ✓✓ = Recommended
- ✓✓✓ = Highly Recommended

*Adjustable with virtual pliers (robot) only
 **Adjustable with virtual pliers (robot) or by hand tools

Space Closure Considerations

Stiffness	Type of Material	Space Closure (1-2mm)	Space Closure (3-4mm)	Archwidth
Low	Nickel Titanium (NiTi)*	✓✓✓ (sliding mechanics)	✓✓ (sliding mechanics with auxiliaries)	✓✓
	Beta Titanium (Beta-Ti)**	✓ (sliding mechanics)	✓✓ (sliding mechanics)	✓✓✓
High	Elgiloy**	✓✓✓ (sliding mechanics)	✓✓✓ (sliding mechanics)	✓

- ✓ = Adequate
- ✓✓ = Recommended
- ✓✓✓ = Highly Recommended

*Adjustable with virtual pliers (robot) only


**Adjustable with virtual pliers (robot) or by hand tools

Identify Orientation Issues for Custom Wires

All of these wires are custom-bent by SureSmile robots located in Richardson, Texas (near Dallas).

When we say the wires are custom-bent precisely to the doctor's prescription, what do we mean?

- The upper wire is different from the lower, of course
- The wire has a top/bottom (left/right) orientation
- The wire has a correct position for the bends relative to the brackets

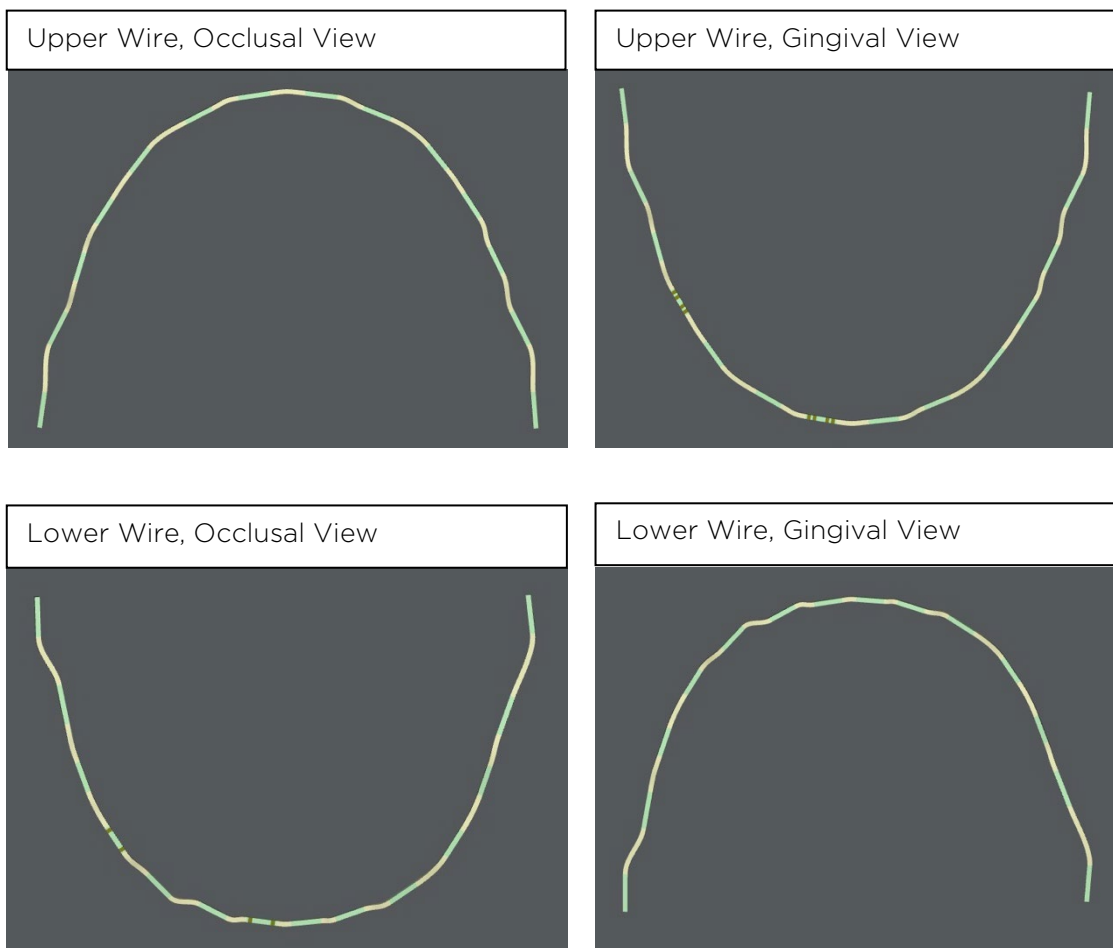

The wire bends are placed between straight segments, which allow the bracket to slide in the direction of the planned movement. That means, you will never position a bend in a slot.

To help you correctly place wires, we have laser markings on the wires. Also, you can compare your inserted wire against the model in the software.

Avoid Bracket Breakage

Do you remember which information SureSmile requires to produce custom wires?



What if a bracket must be rebonded, will that disrupt the design of a custom wire?

It can, especially if more than one bracket must be rebonded. For this reason, we provide a handout to help you educate your patients on how it is especially important not to break brackets when you are a SureSmilepatient. If multiple brackets must be rebonded after the Therapeutic Scan, we recommend taking an Update Scan (extra visit for the patient) and ordering new custom SureSmilewires.

Avoid Wire Breakage


Along this line, you will also want to make sure you are using Weingart or Bird Beak pliers for wire insertion. Because they do not crimp Ni-Ti, they will not damage the prescription.

When you follow our recommended protocol, wire breakage is very rare.



Fully Engage Custom Wires

Because custom wires are designed to deliver their prescription when fully engaged, we recommend steel ligature ties where needed. You cannot achieve the planned movement without full engagement!

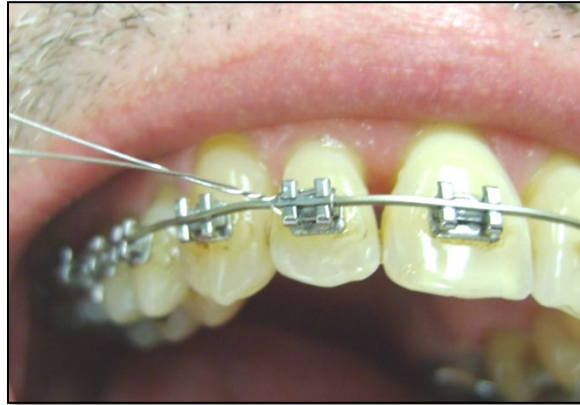

Identify the Purpose of Staged (“Limit”) Wires

To give the doctor complete control over the wire design, the doctor can stage movements by entering a percentage of movement to be achieved. A staging amount can be entered for the entire wire or for each tooth. A value of 100% indicates the wire is fully expressed; a value of 0% indicates the wire is passive.

The most common situations for using this feature are:

- Difficulty of wire engagement
- Ankylosis
- Root resorption
- Periodontally involved teeth
- Low tolerance to discomfort
- Bracket delamination

In orthognathic surgery cases where the surgeon does not want any dental compensations to occur so that he/she can surgically correct a deformity and minimize relapse

Although this is a very useful feature, it is possible to forget to insert the next wire with 100% expression. Or, you may find yourself expecting too much from a wire, forgetting that it contains limits. We will look at some examples later.

Test Your Knowledge

What can you do to increase the flexibility of a NiTi wire to aid insertion?

How much working time do you have once you stop applying the cooling agent?

Does it matter whether custom wires are engaged?

Would you ever position a bend in the bracket slot?

What are staged wires for?

Lesson 2: Checking the Rx Box

Because the custom prescription must be inserted correctly, the prescription box contains a great deal of helpful information.

Objectives

By the end of this section, you will be able to:

- Find labeling to confirm the wire
- Find the recommended first insertion tooth
- Check the wire against the template

Find Labelling

Take a look at the back of the wire prescription box. Extra information on the box helps you determine who this wire was bent for and the type of wire:


- **The patient name and patient number** — ensure this wire is for this patient.
- **Upper or Lower** — which arch is in this box.
- **Labial or Lingual** — type of treatment
- **Material** — such as NiTi
- **Wire Dimensions** — the cross-section, such as .017 x .025

At the foot of the label, the “Wire name” corresponds to the wire name in the software. It is a best practice to name wires in the order of treatment sequence (i.e. Wire 1, 2, 3, 4) and include a description to remind you of the wire’s purpose.

The Dr. Notes section includes a recommendation on which tooth to tie in first. The software makes this determination based on the most maloccluded tooth with the least bend/bracket interferences. In the case pictured at right, the UL5 is recommended. The “MODS SFT” or “SFT” note indicates there is more information in the wire notes in the software about wire modifications.

Moles & Ferri Orthodontics Site ID: 000390001 **SN** 5001104636


Hall, Oliver # H1P9
SO - 000001321287



UL1 UR1
UL2 UR3
UL3 UR4
UL4 UR5
UL5 UR6

Laser markings MUST point to GINGIVAL surface

SN 5001104636



Hall, Oliver
H1P9
4 / Upper / Labial
CuNiTi-NiTi 0.017x0.025 in
(0.432x0.635 mm)

(01) 00856379007252
(11) 210311
(21) 01707695

REF ASY-000110

Symbols: □ symbol indicates laser marking position
⊖ symbol for factory use only

Dr.Notes (1): 4U: 17x25 NiTi
Dr.Notes (2): Insert at UR2
SFT

2/20/2022 11:49:27 AM / NBT03 / 30014H6474
DL-00003332539_031-000390001-HO0000002-0-LI01-W31.CNA

6L: 17x25 NiTi E
H1P9
Hall, Oliver
Prescribed by: Peter Moles
Lbial / Upper / CuNiTi-NiTi 17x25

Scan to receive

Double markings on gingival view of upper wire

Markings

Identify patient

Upper or lower arch

Material

Wire name

Tooth to tie in first


Label visible when stored

Moles & Ferri Orthodontics Site ID: 000390001

SN 5001104637

Hall, Oliver # H1P9

SO - 000001321287

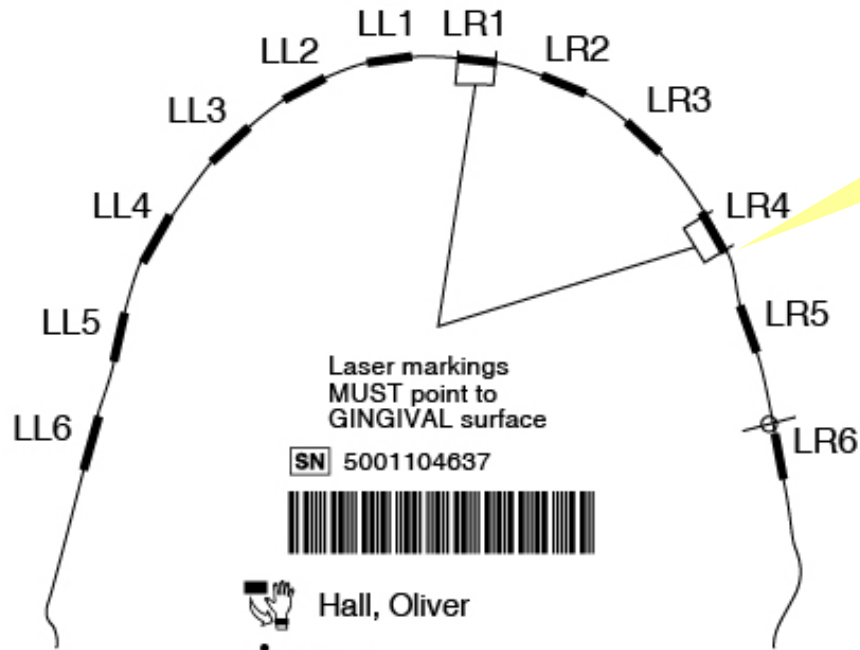

Single marks on occlusal view of lower wire

Upper or lower arch

SN 5001104637



Hall, Oliver

H1P9

2 / Lower / Labial
CuNiTi-NiTi 0.017x0.025 in
(0.432x0.635 mm)

LBOM0005 Rev. 2



(01) 00856379007252
(11) 210311
(21) 01707695

REF ASY-000110

Symbols:

- symbol indicates laser marking position
- symbol for factory use only

Dr.Notes (1): 2L: 17x25 NiTi

Dr.Notes (2): Insert at LL3



2022-02-20 2/20/2022 11:51:37 AM / NBT03 / 30014H6474
DL-00003332540_031-000390001-HO0000002-1-LI01-W31.CNA

Labial / Lower / CuNiTi-NiTi 17x25

Prescribed by: Peter Moles

6L: 17x25 NiTi E

H1P9

Hall, Oliver



Check the Wire Against the Template

Lay the wire that is inside the box on the template. In general, they should match.

Do the bends match?

Do the wire markings match?

Do NOT press the wire down against the template—it is NOT a 2D representation. The template is more like a silhouette of the 3D wire captured during production (similar to how you would see a shadow cast under a spotlight).

If you put pressure on the wire while comparing, you run the risk of:

- Distorting the wire so that it appears the template does not match.
- Trimming the wire too short if the design includes Curve of Spee

Lesson 3: Ordering/Cancelling Wires

Whether or not you are involved in wire ordering, it is helpful to understand how orders will appear in the software.

Objectives

- By the end of this section, you will be able to:
- Order wires individually or as wire sequence
- Remove wire orders
- Submit wire sequence
- Cancel wire in production

NOTE: For more information on the protocols for lingual treatment, see the Lingual Treatment Protocol available as a download on the SureSmile® Help Center.

Doctors follow a wire sequence for SureSmile that depends on:


- Patient type
- Treatment type - labial or lingual
- Slot size

The average number of SureSmile wires ordered per patient is 3-4 wire pairs for labial treatment and more for lingual. Most new SureSmile doctors order more wires as they gain experience with the system. In every case, the doctor will decide the specific wire sequence.

Find Wire Sequence Preferences

Because doctors have a typical wire sequence for patients, SureSmile has a feature to support ordering a sequence all at once. Or, the doctor can order one wire or wire pair at a time.

Store your wire sequences under **Preferences.**

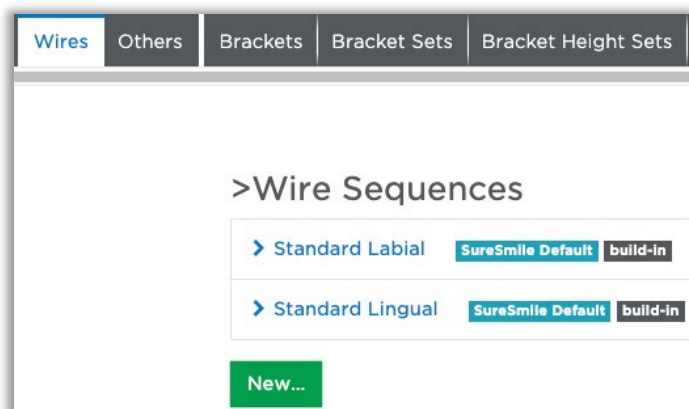


Order Wires Individually or as Wire Sequence


The first wire order can be started from the New menu; however, we strongly recommend starting it after approving the setup. Otherwise, you may lose track of the patient if you approve the setup without starting the Wire Order.

Later in treatment, the doctor will have other options for starting wire orders based on progress.

TIP: If the setup is approved and the user is interrupted before entering the wire order, make sure the wire order is at least created. This will give you a task reminding you to “Order Wire.” Also, notice you have the option to “Remind me later” in the approve dialog, which will also generate this task.

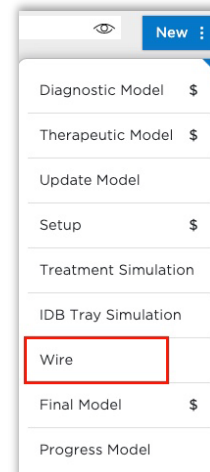

1. Open patient with a plan in review.
2. Click Approve. The dialog opens.

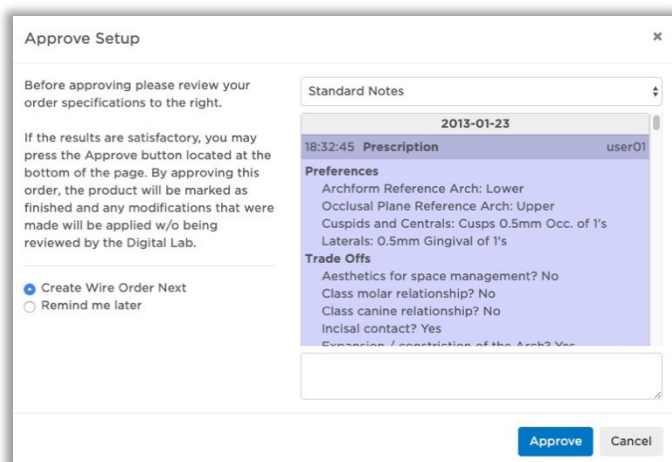

3. Select the “Create wire order next” option and click **Approve**. The “Create New Wire” dialog opens.


Order one wire/pair- Use the Continue button to start a wire order that contains one wire/pair.

Multiple wires/pairs - Use the Apply Wire Sequence button to start a wire order that contains multiple wires/pairs.

4. Click **Apply Wire Sequence** to open the list of choices.

If you wanted to override any of the selections in the sequence, you have the opportunity here to edit it before creating the order.

5. Select the Standard Labial sequence in the drop-down menu.
6. Click **Apply Sequence**. The wire ordering page opens. Notice the page is named “WS 1” in the list.

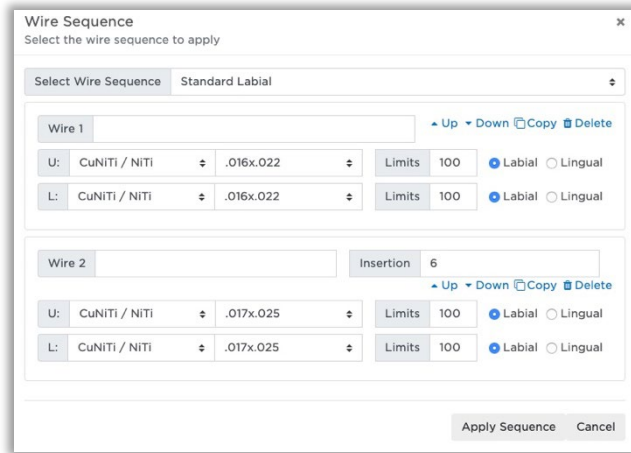

7. Select the **Wire Sequence** tab to see wire design options.

Additional features (e.g. COS, RCOS, Expansion, Constriction) allow the doctor to enhance the effectiveness of the wires, to minimize side effects, and make over-corrections or other anticipated adjustments.

8. Select the **Order** tab to see options for completing the order.

Remove Wire Orders

Notice that when the wire order is not submitted, you have the option to remove it. At this point, no one but you knows about this order so you can remove it without notifying anyone.


Don't forget you need to start the new wire order immediately before you forget about this patient!

Submit Wire Sequence

When you submit the wire sequence order, it will then separate into individual wire orders/pairs.

1. Click Submit Wire Sequence.
2. Click Submit.

Notice the “WS 1” item in the list is replaced by multiple items with the prefix “WS 1” followed by the name you typed in for the wire.

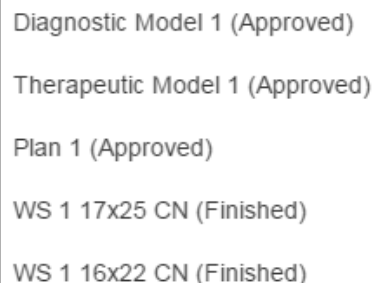


Diagnostic Model 1 (Approved)
Therapeutic Model 1 (Approved)
Plan 1 (Approved)
WS 1 17x25 CN (Finished)
WS 1 16x22 CN (Finished)

Recognize Wire 1

The first set of wires is typically designed directly from the setup, which represents the treatment plan. Wire names in the software default to Wire 1, Wire 2, etc. so usually the name will be Wire 1. If the wires orders are not created in the sequence they will be used, it is a best practice to rename them to reflect the order of insertion. Also, the doctor may type a description into the name as a reminder of the purpose of the wire.

Here are some examples:

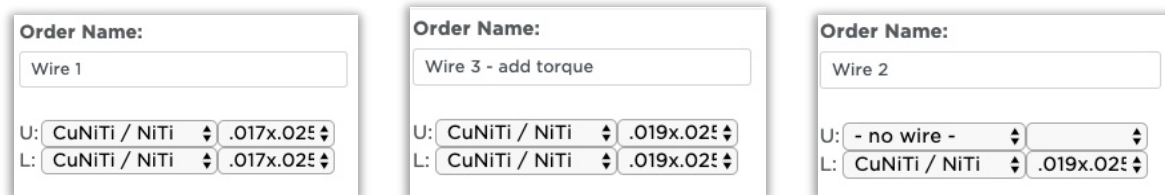

Order Name	U:	L:
Wire 1	CuNiTi / NiTi .017x.025	CuNiTi / NiTi .017x.025
Wire 3 - add torque	CuNiTi / NiTi .019x.025	CuNiTi / NiTi .019x.025
Wire 2	- no wire -	CuNiTi / NiTi .019x.025

Typically, practices start from the smallest cross-section ordered and proceed through treatment up to the largest.

Cancel Wires in Production

If you just ordered a wire (or sequence) and the Digital Lab has not started work on the order yet, you may still cancel an order. However, once work begins, the button becomes unavailable (grayed out). At this point, you will need to call Customer Care if you want to cancel the order.

Test Your Knowledge

How do you start a wire order?

Will you be using the wire sequence feature?

When can you cancel a wire order?

How can you make sure you remember to order wires after checking the setup?

Lesson 4: Opening Finished Wire Orders

At the wire insertion appointment, we recommend opening the wire order in the SureSmile software chairside as a reference. You can access wire orders directly from the patient card.

Objectives

By the end of this section, you will be able to:

- Recognize state information
- Find Wire Insert task or patient card
- View wire details

Recognize States of Wire Orders



A milestone on the timeline shows the status of the custom wire phase of treatment:

- **Estimated** – when the patient record is created (and before wires are ordered), the system sets a “Wire Insertion Dates” milestone to an estimated date by backing up from the debond date. This timeframe is determined by the “Estimated wire to final time” entered in your Preferences for Treatment.
- **Not Ordered** - Once you start a wire order, the name of the order appears in the milestone and the status will be “Not Ordered” until you click the Submit button.
- **Ordered** - the status will be “Ordered” during the entire production process.

During production, your wire order is reviewed and may be adjusted by the Digital Lab. Next, it is sent to manufacturing where it is bent by a robot and checked for quality. Finally, the order is shipped via FedEx. At this point, its status changes to:

- **Finished** - Upon shipping, the wire status in the software is updated from “Ordered” to “Finished” with the current date.

The status of the order remains “Finished,” and now the software begins to track the status of the wire insertion appointment using the same milestone in the timeline. This additional status is visible when you expand the wire order details:

Wire 2		Finished	2011-12-16
U: CuNiTi / NiTi .017x.025		<ul style="list-style-type: none"> Estimated Scheduled Inserted ✓ Not Used 	2011-12-27
L: CuNiTi / NiTi .017x.025			2011-12-27

- **Estimated** - the default status for the wire insertion date. Corresponds to a target date for the insertion appointment defaulted to 2 weeks after the setup creation date, unless an interval was specified in a wire sequence order.
- **Scheduled** - should correspond to the actual date of the wire insertion appointment. You can set this status to indicate that you checked that the wires have arrived.
- **Inserted** - set this status the day of the wire insertion appointment, with the corresponding date.
- **Not Used**- set this status the day of the wire insertion appointment when you realize this wire will not be needed.

The milestone in the timeline displays the most recent wire activity.

Find “Insert Wire” Task on Day of Appointment


If you have set the estimated/scheduled wire insertion date on the timeline, an “Insert Wire” task appears on your task page. It will display if the date is tomorrow, today, or in the past to help you quickly access the patient’s record.

1. Sort on the **Wire Insertion Date** column, if you would like the most recent dates to appear at the top of the list.
2. To further shorten the list, you can also filter by **Show Reminder Tasks**.
3. For case OJ07, click the clock icon to display the patient card.

Find Patient Card on Day of Appointment

If for some reason you have not entered the appointment date to provide an “Insert Wire” task, you can go to the Patient tab and find the patient in a search.

1. On the clinic overview, click **Patients** to list your patient cards.
2. Find case Z8M7.
3. Click the clock ⌚ to expand the timeline.

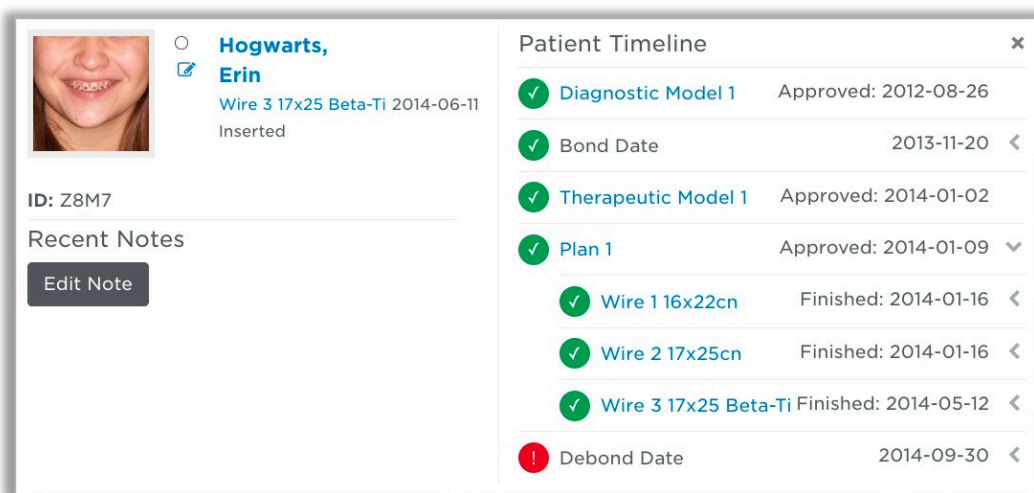

View Wire Orders in the Patient Card


You can see an overview of the wires ordered for a patient in the drop-down list on the patient card.

Click the drop-down arrow for each wire order to see the date information.


Select **Wire 1** in the timeline to open the corresponding wire order. Don't forget to right-click to open it in a new tab.

TIP: You cannot see the wire cross-section in this drop-down unless you have typed the size into the name of the wire. If needed, you can open the patient record to see the cross-section in the full timeline.

If you have several wire insertion appointments, you can open each patient record now.

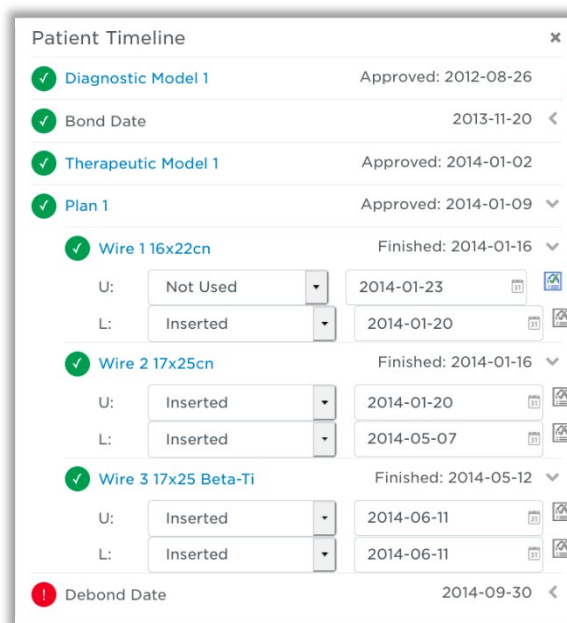

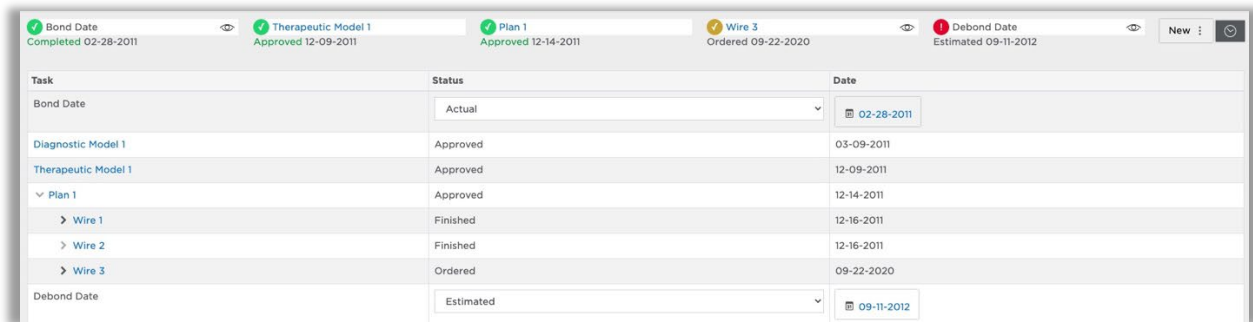

View Wire Orders in the Full Timeline



Task	Status	Date
Bond Date	Actual	02-28-2011
Diagnostic Model 1	Approved	03-09-2011
Therapeutic Model 1	Approved	12-09-2011
Plan 1	Approved	12-14-2011
Wire 1	Finished	12-16-2011
Wire 2	Finished	12-16-2011
Wire 3	Ordered	09-22-2020
Debond Date	Estimated	09-11-2012

Typically, you will install Wire 1 first. However, if you need to check the cross-sections to be sure, look at the full timeline after you open the case.

1. Click the clock to display the timeline. 
2. Click the large clock to drop-down the full timeline details. 

Now, you can see all wire details:

- Wire order name
 - Status of the wire order
 - Date of the order
 - Wire cross-sections
 - Date of wire insertion appointment
 - Status of wire insertion
3. When you are sure of the wire to open, select it in the list.

Test Your Knowledge

When is a wire order status marked as Finished?

How long does it take wires to ship to your office?

How will you know which wire is the first wire?

When will the insert wire task appear on your Tasks page?

Who will set the status of wires to Inserted/Not used?

Where can you view information about the wire orders?

Lesson 5: Navigating the Wire Design Workspace

The Wire Order page has the same palettes and timeline across the top, but the rest of the user interface has a different layout.

Objectives

By the end of this section, you will be able to:


- Open wire order
- Display Model Parts
- Recognize Limit Wires

Open Wire Order

The wire order page shows two views of the wire design. On the left, the wire is shown in its activated state as it would appear when tied into the malocclusion. We call this wire (shown in green) the active wire.

On the right, the wire is shown in its expressed state as it would appear when the setup is achieved. We call this wire the full expression wire.

These are two different views of the same wire, current and future

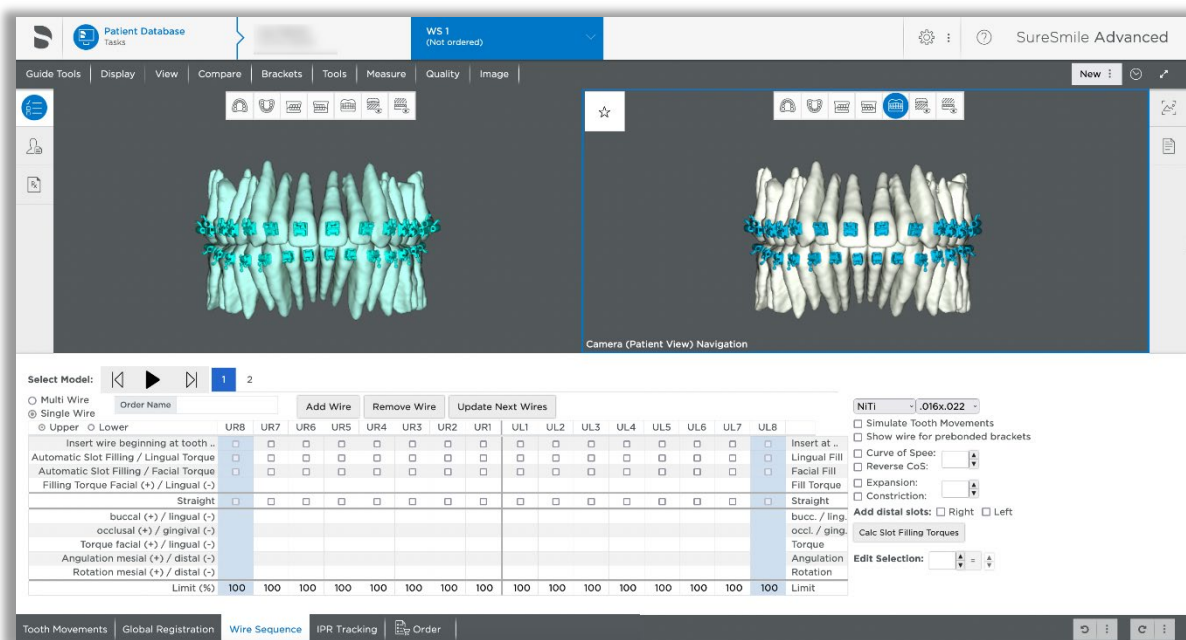

At the bottom of the page, you will see some tabs:

- **Order tab** - shows information on the wire box and includes a copy button in case the doctor wants to order another wire based on this design.
- **Wire tab** - shows wire design details. Notice a toggle to switch between the upper and lower wire. The first wire pair might not have any overrides.
- **IPR Tracking** - displays IPR planned in the setup and allows you to track IPR applied at the appointment. We recommend you also continue to track IPR in your PMS charting.

Order Name: Wire 1

Start Reference Model: Therapeutic Model 1

Wire Order: Create Copy

Referenced Scan: Therapeutic Model 1

Base Model: Plan 1

Wire Label:

Upper	INSERT AT UL4
Lower	INSERT AT LL4

U: CuNiTi / NiTi .017x.025

L: CuNiTi / NiTi .017x.025

Order

Recognize Limit Wires

Remember the doctor can order staged wires (also called limit wires or percentage wires). You will see percentages entered on the Wires tab. It is a best practice to indicate the % in the wire name to help you continue to progress to the 100% wire.

% typed into name as reminder

Limited movements are visible in 3D model

Limit % shown on last row of Wire tab (select Upper or Lower)

Order Name	UR8	UR7	UR6	UR5	UR4	UR3	UR2	UR1	UL1	UL2	UL3	UL4	UL5	UL6	UL7	UL8	Limit (%)
Insert wire beginning at tooth...	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	100
Automatic Slot Filling / Lingual Torque	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	100
Automatic Slot Filling / Facial Torque	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	100
Filling Torque Facial (+) / Lingual (-)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	100
Straight	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	100
buccal (+) / lingual (-)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Limit
occlusal (+) / gingival (-)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Limit
Torque facial (+) / lingual (-)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Limit
Angulation mesial (+) / distal (-)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Limit
Rotation mesial (+) / distal (-)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Limit

On the wire order page, notice the tooth displacements in the 3D model match the limit values of the wire to provide a simulation of movement to be achieved by the wire.

Display Notes

By default, the notes on the wire order will display when you open notes. However, you can switch to the plan notes to see warnings from the setup technician on interferences.

1. Click the notes icon  in the upper-right corner of the screen.

Notes


Patient Notes

Jen Brice, 2019-05-21 at 11:22:06AM
Midline correction needed

Add note here...

+ Save Note


2. Check the Practice Internal Notes. You can use this feature to communicate between the doctor and team about any aspect of a patient's treatment.
3. Click the **Practice Internal Notes** drop-down in the notes window and select **Product Notes**.
4. Click the **Wire 1** drop-down and select **Plan 1**. (The names are specific to the patient.) The notes from the plan display, including a note about bracket interferences.
5. Close the window by clicking the x in the corner.

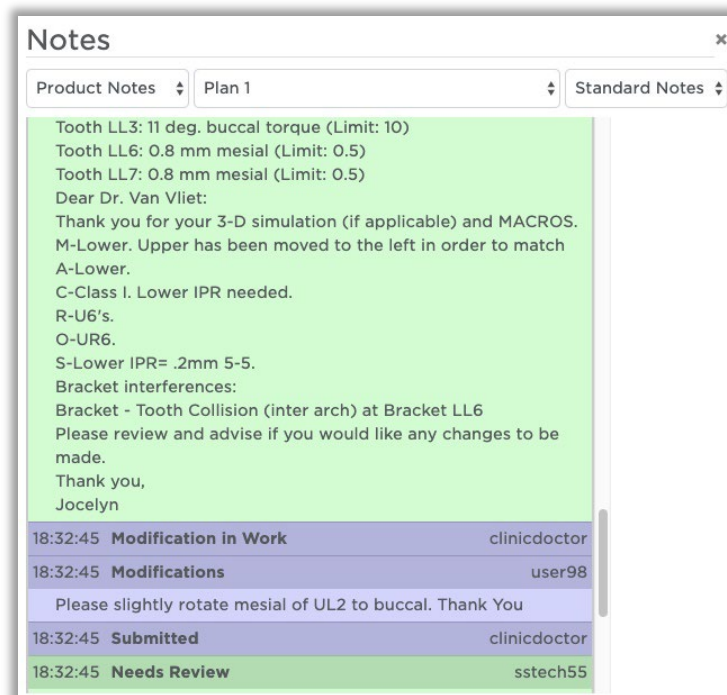

Display Model Parts

You can manipulate the 3D models to view the information you need.

The **Display** palette shows or hides the arches.

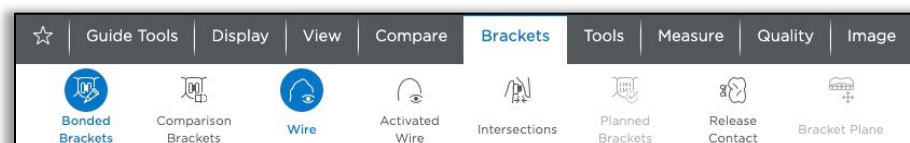

The Brackets palette shows/hides the brackets and wires.

What do these parts represent?

▶ Light blue teeth -

▶ Light blue brackets -

▶ Green wire -

▶ White teeth -

▶ Dark blue brackets -

▶ Gold/white wire -

Add these icons to your Favorites palette to speed up displaying what you need during the wire insertion appointment.

Apply Your Knowledge

While logged in under your username, build a favorites palette with icons for the wire insertion appointment.

Lesson 6: Finding/Recording IPR

The IPR is calculated from the approved setup, but you do not need to open the Plan to see IPR values. These values are available in a couple of other areas of the software for your convenience during the wire insertion appointment.

Objectives

By the end of this section, you will be able to:

- Find planned IPR
- Enter IPR
- Generate an IPR report
- Recognize an IPR reminder task

Find Planned IPR

Since the setup represents the target for a patient’s treatment, the IPR needed to achieve the final tooth positions is recorded with the setup (a.k.a. the Plan). But it is also visible in other locations for easy access during wire insertion:

- IPR Tracking tab
- IPR Report

	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Planned				-0.3	-0.3	-0.3	-0.3	-0.4	-0.2							
Applied																
Completed	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Completed	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Applied																
Planned				-0.2	-0.2	-0.2	-0.4	-0.2	-0.2	-0.3	-0.3					
	32	31	30	29	28	27	26	25	24	23	22	21	20	19	18	17

Edit Selection: =

Tooth Movements |
 Staged Timeline |
 IPR Tracking |
 Order

Enter IPR Chairside


For the fastest method to determine if IPR is planned, select the IPR Tracking tab. This tab is available in every wire order (and also in the setup). If there are no values in the “Planned” rows for the upper or lower wires, simply return to the Order tab.

- If IPR is completed for an area, click the checkbox in the “Completed” row. (Remember SureSmile reports IPR for the mesial side of one tooth, but most doctors apply half the value to the two adjacent teeth.)
- If IPR is partially applied, type in the amount in the “Applied” row.

Generate IPR Report

If you are not paperless, you can generate an IPR report to print. Since this report is specific to a patient, it is listed in the Reports drop-down on the patient overview page.

1. Go to the patient overview.
2. Click **Reports** and select **Patient IPR Report**. A dialog opens to confirm the plan.
3. Select the appropriate plan and click **Apply**. The report generates and displays in a new tab.
4. Use your browser to print the report. (You may need to right-click for print options.)
5. When you are finished, close the tab with the report and close the dialog.

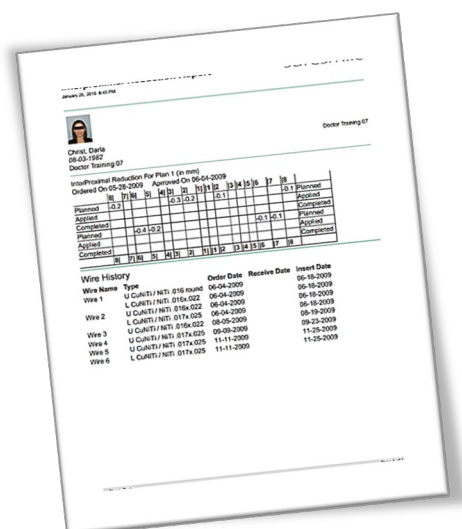

Complete an IPR Reminder Task

The software will create a reminder to apply IPR under these conditions:

- At least one wire status is set to “Inserted.”
- Total planned IPR has not yet been indicated as completed on the IPR Tracking tab.

A rectangular button with a light gray border and a white background. The text "Complete IPR" is in a dark gray font, followed by a small blue circular icon containing a white right-pointing arrow.

When you have completed the IPR planned for a patient, follow these steps to clear the task.

1. If you do not see the Complete IPR task, click the **Show...Tasks** filter and choose **Reminder**. Or, you can enter the patient ID in the quick search box.
2. Click the 🕒 (clock) at left to expand the patient's timeline and select any wire order to open it.
3. In any wire order, select the **IPR Tracking** tab.
4. In the Completed row, check the boxes to indicate you have finished applying all IPR for the case.

Lesson 7: Getting Ready for Wire Appointments

Since wires are custom-made for treatment efficiency, add some steps to your office routine to ensure that wires have arrived in time for the wire insertion appointment.

Objectives

- By the end of this section, you will be able to:
- Find wire information
- Check Appointments vs. Received Wire Shipments
- Set Estimated to Scheduled Dates
- Print IPR report

Wire Management - An Office Tour


Brenda Zullo, Dr. Moravec's award-winning SureSmile Coordinator, shares her system for tracking and storing SureSmile wire prescriptions to ensure patients receive the right wires at the right time. These tips may or may not be useful for your practice; her system is merely an example of how one office manages wire prescriptions throughout the SureSmile Process.

[Click here for tour](#)

Track/Receive Wire Shipments

As wire packages arrive, use the **Shipments** page to enter them as received and then store them. If you have multiple sites, the Shipments page can also help you keep track of moving wires to satellite offices.

The SureSmile wire tracking feature is located on the clinic overview under the **Shipments** tab. Who will be responsible for this activity? We will meet separately about the functionality of this area (see next module).

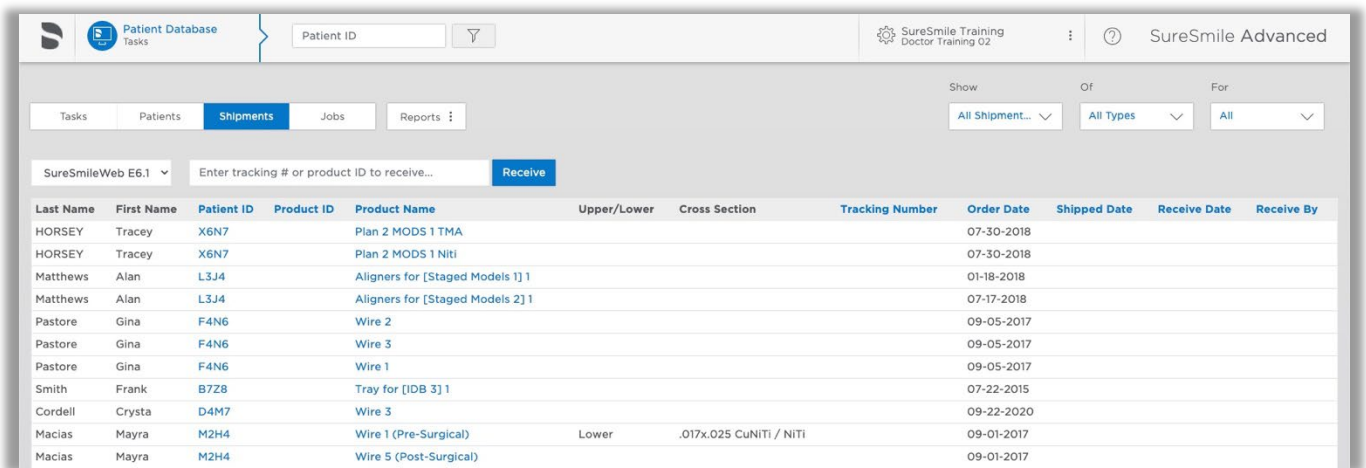

Last Name	First Name	Patient ID	Product ID	Product Name	Upper/Lower	Cross Section	Tracking Number	Order Date	Shipped Date	Receive Date	Receive By
HORSEY	Tracey	X6N7		Plan 2 MODS 1 TMA				07-30-2018			
HORSEY	Tracey	X6N7		Plan 2 MODS 1 Niti				07-30-2018			
Matthews	Alan	L3J4		Aligners for [Staged Models 1] 1				01-18-2018			
Matthews	Alan	L3J4		Aligners for [Staged Models 2] 1				07-17-2018			
Pastore	Gina	F4N6		Wire 2				09-05-2017			
Pastore	Gina	F4N6		Wire 3				09-05-2017			
Pastore	Gina	F4N6		Wire 1				09-05-2017			
Smith	Frank	B7Z8		Tray for [IDB 3] 1				07-22-2015			
Cordell	Crysta	D4M7		Wire 3				09-22-2020			
Macias	Mayra	M2H4		Wire 1 (Pre-Surgical)	Lower	.017x.025 CuNITI / NITI		09-01-2017			
Macias	Mayra	M2H4		Wire 5 (Post-Surgical)				09-01-2017			

Check Appointments vs. Received Wire Shipments

Before you can insert SureSmile wires, you must have received them. Since it is extremely inconvenient for all if the patient arrives and the wires haven't, every week your office will need to check on the wire appointments that are about a week away. If you find out you have not received the SureSmile wires for the appointments, there is still time to figure out why and make sure they arrive in time. Usually one person, such as an assistant or the SureSmile Coordinator, takes care of this task - who will it be in your office?

Decide on a regular day each week to check on wires. On that day, pull a report from your PMS on the following week's SureSmile wire insertion appointments. You should set up a procedure code for SureSmile wire insertion (this appointment takes about 30 minutes). Once you have the report, open SureSmile and view the Shipments page. Lookup each patient to see if the wires have arrived or are expected within the next week.

Set Estimated to Scheduled Dates

To help you keep track of the patients you have verified, change the date for the wire insertion on the timeline from “Estimated” to “Scheduled” and use the correct date. As a result, a task will appear the day of the appointment to assist you in finding the patient’s record.

If your office is paperless, you are ready for the appointment!




Print IPR Reports (if not paperless)

If you are NOT paperless, this is a good time to produce IPR reports for the doctor. You will need to open each SureSmile patient record to generate the report.

- If the report has any values (it will run even if no IPR is planned), then print it.
- Once you have printed all the reports for your upcoming appointments, go to wire storage and attach the report to its prescription package. You can simply rubber band the report around the package.
- Leave the wires, stored alphabetically by patient last name, until the day of the appointment.

Interproximal Reduction Report suresmile®
 January 26, 2015 8:45 PM

 **Christ, Darla** Doctor Training 07
 08-03-1982
 Doctor Training 07

Interproximal Reduction For Plan 1 (6 mm)
 Ordered On 05-28-2009 Approved On 06-04-2009

	6	7	8	9	4	3	2	1	1	2	3	4	5	6	7	8
Planned	-0.2															
Applied																
Completed																
Planned																
Applied																
Completed																

Wire History

Wire Name	Type	Order Date	Receive Date	Insert Date
Wire 1	U CoNiTi / NITI .016 round	06-04-2009		06-18-2009
	L CoNiTi / NITI .016x.022	06-04-2009		06-18-2009
Wire 2	U CoNiTi / NITI .016x.022	06-04-2009		06-18-2009
	L CoNiTi / NITI .017x.025	06-04-2009		06-18-2009
Wire 3	U CoNiTi / NITI .016x.022	09-05-2009		08-19-2009
	L CoNiTi / NITI .017x.025	09-09-2009		09-23-2009
Wire 4	U CoNiTi / NITI .017x.025	11-11-2009		11-25-2009
	L CoNiTi / NITI .017x.025	11-11-2009		11-25-2009

Christ, Darla Page 1 of 1


Lesson 8: Following the Wire Insert Protocol

SureSmile wires are custom-made to the doctor's prescription and must be inserted correctly to achieve planned results. To help you correctly place wires, we recommend referring to the wire order in the SureSmile software during the procedure. If you do not have computers chairside, you can use the doctor's laptop with a wireless connection to the network.

The wire insertion procedure for the first wire prescription follows these steps:

- The first step is to set up for the appointment, which includes opening the wire order in the software and identifying the first set of wires (if the doctor has ordered multiple prescriptions).
- Next, get the first wire out, check the prescription and get it ready.
- Referring to the markings, place the wire.
- Check the software to see how the wire should look when positioned properly.
- Your office may already be accustomed to fully engaging wires. This is an especially important step for SureSmile wires because the plan always reflects fully engaged wires.
- As usual, the doctor checks the wires and bite.
- Then the doctor refers to the software to make sure expected movements are achievable.
- Please take 2 seconds to update the wire insertion information on the timeline before closing the SureSmile patient record.
- Finally, don't forget to celebrate this milestone in the patient's treatment!

Now, let's look at these steps in more detail.

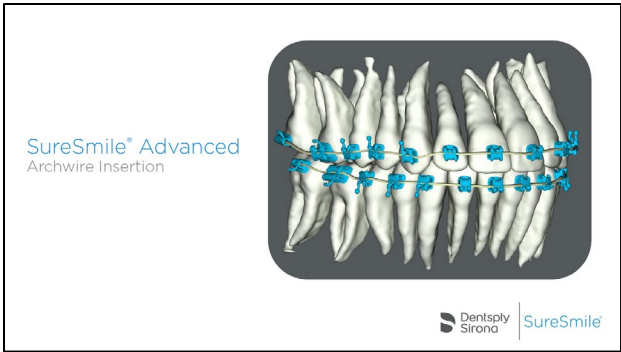

Objectives

- Find wire tracking feature
- Prepare for the day's wire insertion appointments
- Recognize wire markings and instructions
- Open plans & wire orders in SureSmile®
- Find doctor's plan for IPR
- View 3D model, brackets & wires

©2022 Dentsply Sirona Orthodontics Inc. 284-C046888-2303-000001-Cent-3-Inst-04_402-18-000001-10-10202-All-Right-Reserved-0

First Wire Insertion Support Visit

- Develop a routine to prepare for the day's wire insertion appointments
- Check wires and cut to correct length
- Position bends correctly and fully engage wires
- Provide patient instructions, reinforce benefits of SureSmile® treatment

©2022 Dentsply Sirona Orthodontics Inc. 284-C046888-2303-000001-Cent-3-Inst-04_402-18-000001-10-10202-All-Right-Reserved-0

Appointment Preparation

©2022 Dentsply Sirona Orthodontics Inc. 284-C046888-2303-000001-Cent-3-Inst-04_402-18-000001-10-10202-All-Right-Reserved-0

Each week, check on upcoming appointments

Who will do this job?

- Practice Management System report – next week's SureSmile® wire appointments (coded)
- Wire Tracking – wires received?
- (Paperless – no other action needed until day of appointment)

©2022 Dentsply Sirona Orthodontics Inc. 284-C046888-2303-000001-Cent-3-Inst-04_402-18-000001-10-10202-All-Right-Reserved-0

If you do NOT have chairside computers...

- Open the SureSmile® patient record
- Generate IPR report (*select appropriate Plan*)
- Print and attach to prescription package or file in chart

©2022 Dentsply Sirona Orthodontics Inc. 284-C046888-2303-000001-Cent-3-Inst-04_402-18-000001-10-10202-All-Right-Reserved-0

Insertion Supplies

- Add patient's SureSmile® wires as you set up tray
(If multiple sets – take all)
- Weingart or Bird Beak pliers
- Method to chill wires
(plus cotton roll or Q-tips)

©2022 Dentsply Sirona Orthodontics Inc. 284-C046888-2303-000001-Cent-3-Inst-04_402-18-000001-10-10202-All-Right-Reserved-0

The Initial Wire Insertion Procedure

©2022 Dentsply Sirona Orthodontics Inc. 284-C046888-2303-000001-Cent-3-Inst-04_402-18-000001-10-10202-All-Right-Reserved-0

Overview of Chairside Procedure

- Identify first prescription wire for insertion
- Review information in software with doctor
- Resolve interferences / perform IPR
- Insert wire
- Refer to 3D model for proper positioning
- Fully engage wire
- Doctor checks wires and bite
- Update timeline information
- Instruct patient & celebrate!



©2022 SureSmile Group, Inc. 361 Coakley, 2300 Campbell Creek Blvd, Ok, 400, Richardson, TX 75082. All Rights Reserved.

Identify First Wire for Insertion



Wire sequencing - Which wire is first?

- % wires
- Then generally small to large cross sections

Select wires and confirm:

- Patient Name
- Wire number/name
- Wire material/cross-section

©2022 SureSmile Group, Inc. 361 Coakley, 2300 Campbell Creek Blvd, Ok, 400, Richardson, TX 75082. All Rights Reserved.

Review with Doctor

Open Plan


- Check MACROS area to look for tooth/bracket/wire collisions

Open Practice Internal Notes

- Check for notes from doctor regarding IPR/use of rubber bands or other auxiliaries

Check for IPR

- Open tab on wire page

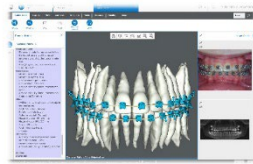

©2022 SureSmile Group, Inc. 361 Coakley, 2300 Campbell Creek Blvd, Ok, 400, Richardson, TX 75082. All Rights Reserved.

Review with Doctor

Open wire:

New 1


Select wire you are inserting:

Wire 1



©2022 SureSmile Group, Inc. 361 Coakley, 2300 Campbell Creek Blvd, Ok, 400, Richardson, TX 75082. All Rights Reserved.

Review with Doctor

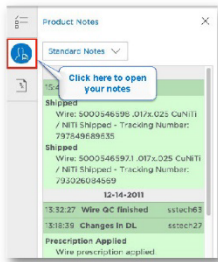

TIP: You can open multiple tabs in your browser for your patient and enjoy the ease of switching views.

Simply right click and selection "open in new tab".


©2022 SureSmile Group, Inc. 361 Coakley, 2300 Campbell Creek Blvd, Ok, 400, Richardson, TX 75082. All Rights Reserved.

Resolve interferences/perform IPR



©2022 SureSmile Group, Inc. 361 Coakley, 2300 Campbell Creek Blvd, Ok, 400, Richardson, TX 75082. All Rights Reserved.

Wires – Essentials, Orientation and Trimming Ends

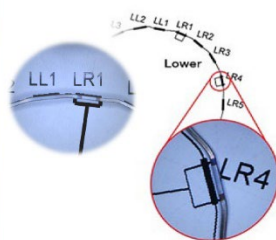


Lower Wire Markings

- Patient right
- Occlusal side of wire
- LR1 and LR4 (if present)
- Mesial/Distal ends of straight segment

©2022 SureSmile Group, Inc. 361 Coakley, 2300 Campbell Creek Blvd, Ok, 400, Richardson, TX 75082. All Rights Reserved.

Wires – Essentials, Orientation and Trimming Ends

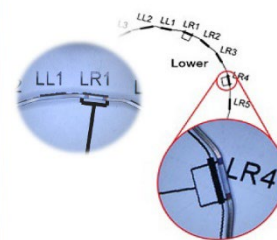

Upper Wire Markings


- Patient right
- Gingival side of wire
- UR1 and UR4 (if present)
- Double marks

©2022 SureSmile Group, Inc. 361 Coakley, 2300 Campbell Creek Blvd, Ok, 400, Richardson, TX 75082. All Rights Reserved.

Wires – Essentials, Orientation and Trimming Ends


Therapeutic Model:

- teeth are blue
- brackets are light blue

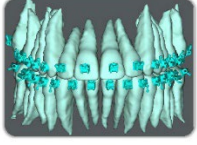

Setup/Plan:

- teeth are white
- brackets are dark blue



©2022 Suresmile Dental Orthodontics, Inc. 256-Collins St. 28300 Coastal Creek East, Ok. 480, Oklahoma, TX 73102. All Rights Reserved. 17

Wires – Essentials, Orientation and Trimming Ends

Archwire Essentials

- Wires designed with straight (unbent) and bent wire segments. Brackets should always fit in straight segments.
- To aid in positioning use Therapeutic Model (blue) teeth with display wire
- Markings on the wire reflect planned mesial/distal movements from the Therapeutic Model to the Plan
- Engage wire with markings at near therapeutic positions in the 3D Model





©2022 Suresmile Dental Orthodontics, Inc. 256-Collins St. 28300 Coastal Creek East, Ok. 480, Oklahoma, TX 73102. All Rights Reserved. 18

Wires – Essentials, Orientation and Trimming Ends

Open prescription box

- Match patient name and wire #
- Verify size or cross-section and material
- Review Doctor's notes on wire template and in software
- Use recommended insertion point (place first tie on insertion tooth then alternate)

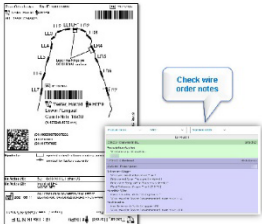

©2022 Suresmile Dental Orthodontics, Inc. 256-Collins St. 28300 Coastal Creek East, Ok. 480, Oklahoma, TX 73102. All Rights Reserved. 19


Wires – Essentials, Orientation and Trimming Ends

Wire match prescription box?

Line up wire with template

- Do the bends match?
- Do the laser markings match?



If it doesn't match, do not use!



©2022 Suresmile Dental Orthodontics, Inc. 256-Collins St. 28300 Coastal Creek East, Ok. 480, Oklahoma, TX 73102. All Rights Reserved. 20

Wires – Essentials, Orientation and Trimming Ends


- Mark wire distal of the last bend in preparation to cut distal ends

©2022 Suresmile Dental Orthodontics, Inc. 256-Collins St. 28300 Coastal Creek East, Ok. 480, Oklahoma, TX 73102. All Rights Reserved. 21

Wires – Essentials, Orientation and Trimming Ends

- Cut the distal ends of the wire at a 45° angle to avoid fraying
- Minimum of 5mm of wire distal to the 7's until the wire is seated (to allow play)

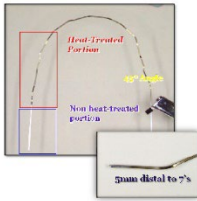



©2022 Suresmile Dental Orthodontics, Inc. 256-Collins St. 28300 Coastal Creek East, Ok. 480, Oklahoma, TX 73102. All Rights Reserved. 22

Insert Wire

Use laser markings to check your initial position

- Upper wire markings face gingival
- Lower wire markings face occlusal

Engage wire with mark near therapeutic bracket location with room to move to approved setup bracket location

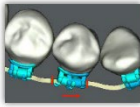


©2022 Suresmile Dental Orthodontics, Inc. 256-Collins St. 28300 Coastal Creek East, Ok. 480, Oklahoma, TX 73102. All Rights Reserved. 23

Refer to 3D Model for Proper Positioning

Refer to software for positioning

- Therapeutic Model
- Passive Wire


Bends are NEVER in slot
Brackets ALWAYS on straight segments

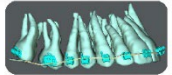©2022 Suresmile Dental Orthodontics, Inc. 256-Collins St. 28300 Coastal Creek East, Ok. 480, Oklahoma, TX 73102. All Rights Reserved. 24

Fully Engage Wire

- ☞ Tie in first tooth as recommended, alternate sides
- ☞ Check for full engagement

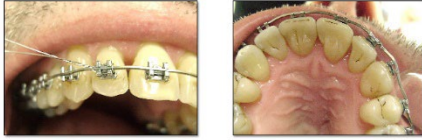

©2022 Dentsply Sirona Orthodontics Inc. 284-C006166 2303 Complete Care 8-04-04. All Rights Reserved.

Fully Engage Wire

Use a chilling option if necessary

- ☞ Cold spray with cotton swab
- ☞ Polar ice stick



©2022 Dentsply Sirona Orthodontics Inc. 284-C006166 2303 Complete Care 8-04-04. All Rights Reserved.

Doctor Checks Wires and Bite

- ☞ Alert doctor of any interferences




Tip: For visual of interferences click on "check bracket and wire interferences with teeth" icon



©2022 Dentsply Sirona Orthodontics Inc. 284-C006166 2303 Complete Care 8-04-04. All Rights Reserved.

Update Timeline Information

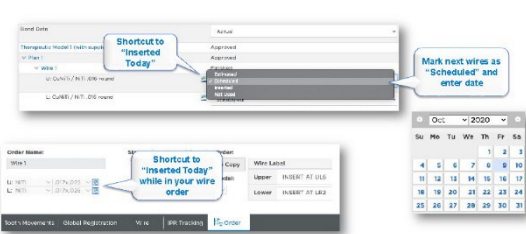

©2022 Dentsply Sirona Orthodontics Inc. 284-C006166 2303 Complete Care 8-04-04. All Rights Reserved.

Instruct Patient and Celebrate

- ☞ Patient instructions
- ☞ Make it special – celebrate!
- ☞ Ideas?




Dentsply Sirona | SureSmile®

©2022 Dentsply Sirona Orthodontics Inc. 284-C006166 2303 Complete Care 8-04-04. All Rights Reserved.

Review of Chairside Procedure

Main steps of procedure

- Identify first wire for insertion
- Review information in software with doctor
- Doctor resolve interferences, perform IPR as planned
- Insert wire
- Refer to 3D model for proper positioning
- Fully engage wire
- Doctor checks wires and bite
- Update timeline information
- Instruct patient & celebrate!

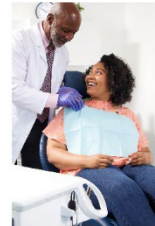

©2022 Dentsply Sirona Orthodontics Inc. 284-C006166 2303 Complete Care 8-04-04. All Rights Reserved.

Helpful Tutorials

Easy access to online Help Center and SureSmileU



©2022 Dentsply Sirona Orthodontics Inc. 284-C006166 2303 Complete Care 8-04-04. All Rights Reserved.

How To: Use Wire Tracking to Find Orders

Scan bar code to quickly check in wires


Barcode Verification

SN: 1283-123-3245-09



©2022 Dentsply Sirona Orthodontics Inc. 284-C006166 2303 Complete Care 8-04-04. All Rights Reserved.

Next Wire Visit Objectives

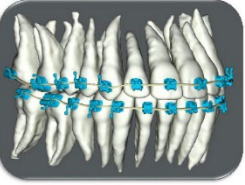

Follow-up wire check appointment:

- ☒ Evaluate wires at follow-up appointments
- ☒ Enter doctor's wire modifications
- ☒ Use of tools for modifications

©2022 Dentsply Sirona Orthodontics Inc. USA. Dentsply, SureSmile, 3D30 Complete Care, 3M, 400, Robinson, TX 75087. All Rights Reserved.

SureSmile® | Dentsply Sirona

©2022 Dentsply Sirona Orthodontics Inc. USA. Dentsply, SureSmile, 3D30 Complete Care, 3M, 400, Robinson, TX 75087. All Rights Reserved.
2021 PHE01-20219 Rev. 01

Discussion

What are the critical steps in SureSmile wire insertion?

How do you plan to celebrate at the initial wire insertion appointment?

Lesson 9: Evaluating Treatment Progress

We recommend allowing the SureSmile wires about 8-10 weeks to fully express. If treatment is staged, you can change to the next wires as needed. Otherwise, if the patient has not achieved the desired outcome, you can open a checklist to guide you through the evaluation of treatment progress. The checklist is located in the software inside the wire order area and includes features to assist you in ordering new wires.

Objectives

By the end of this section, you will be able to:

- Open progress evaluation checklist
- Recognize wires ordered from a sim
- Recognize compensation wires
- View overrides

Open the Treatment Progress Evaluation Checklist


1. Go to the Patients tab and open the wire order corresponding to the currently inserted wires.

2. Click the  icon in the upper-left corner to open the checklist.

NOTE: Do NOT refer to the checklist during [initial](#) wire insertion. The purpose of this checklist is to evaluate treatment progress of SureSmile wires several weeks later.

3. Follow the instructions in the checklist to evaluate wires. You are guided to:

- Confirm wires are inserted correctly
- Confirm the setup has been achieved. If you are not satisfied with the outcome, your options are to create a new wire based on a sim or copy the current wire and modify the design.
- Check for and resolve interferences
- Create a progress model to order a compensation wire

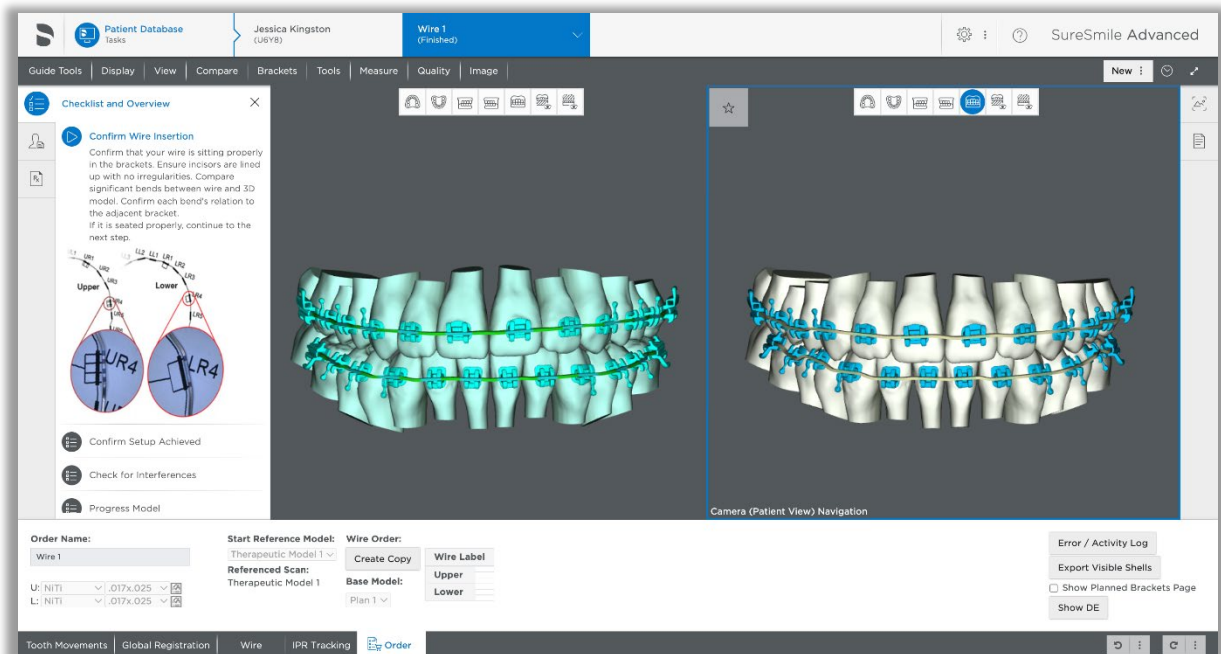


View Overrides

Following the initial wires designed from the setup, your doctor may order more wires with additional modifications for over-correction or compensation, etc. These modifications are also called “overrides” in the SureSmile software.

To view overrides:

1. Open the wire order or switch to the appropriate order using the drop-down list at the top of the treatment planning area.
2. Select the **Shipments** tab.
3. Click the **Upper** or **Lower** option.

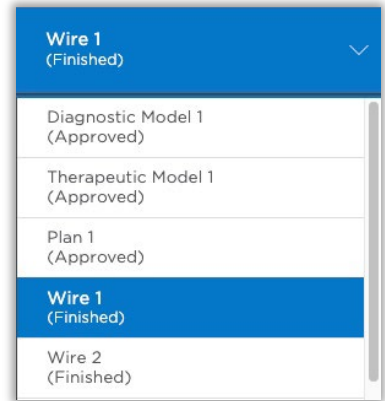

● Upper ○ Lower	UR8	UR7	UR6	UR5	UR4	UR3	UR2	UR1	UL1	UL2	UL3	UL4	UL5	UL6	UL7	UL8	
Insert wire beginning at tooth ..	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Insert at ..
Automatic Slot Filling / Lingual Torque	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Lingual Fill
Automatic Slot Filling / Facial Torque	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Facial Fill
Filling Torque Facial (+) / Lingual (-)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Fill Torque
Straight	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Straight
buccal (+) / lingual (-)																	bucc. / ling.
occlusal (+) / gingival (-)																	occl. / ging.
Torque facial (+) / lingual (-)																	Torque
Angulation mesial (+) / distal (-)																	Angulation
Rotation mesial (+) / distal (-)																	Rotation
Limit (%)	100	100	100	100	100	100	100	100	100	100	100	100	100	100	100	100	Limit

NITI | .017x.025

Simulate Tooth Movements

Show wire for prebonded brackets

Curve of Spee: ▲ ▼

Reverse CoS: ▲ ▼

Expansion: ▲ ▼

Constriction: ▲ ▼

Add distal slots: Right Left

Calc Slot Filling Torques

Edit Selection: ▲ ▼ = ▲ ▼

Tooth Movements |
 Global Registration |
 Wire |
 IPR Tracking |
 Order

Recognize Common Strategies

You will notice your doctor ordering wires based on some common strategies for achieving treatment goals:

Goal	Strategy
Finish by a deadline	Order multiple cross-sections and start with the largest cross-section that can be inserted. (In this case, the smallest wire is not used and can be discarded.)
Allow for further alignment, minimize bracket placement errors	Order undersized wire.
Close space	Select straight segments in the wire design.
Move teeth conservatively due to dental health or discomfort	Enter a "Limit %" to reduce forces from 100% (full expression) to a lower percentage. NOTE: The doctor will usually order the full expression wire at the same time as limit wires. NOTE: Limit wires may also be referred to as "staged wires."
Compensate for auxiliaries, over-correct, etc.	Add modifications ("overrides") to the wire design.

As your doctor gains more experience in working with SureSmile®, you will see additional strategies such as ordering overrides on the first set of wires.

Tracking Wire Shipments

Viewing SureSmile Shipments

The SureSmile software includes a Shipments tracking page to help you ensure that a patient's archwires have arrived at your office in time for the insertion appointment. You may also visit this page to track other shipped items such as IDB trays or printed models. Use the same features to check on all SureSmile shipments.

Last Name	First Name	Patient ID	Product ID	Product Name	Upper/Lower	Cross Section	Tracking Number	Order Date	Shipped Date	Receive Date	Receive By
Skomerza	Mary	S4H2	183102	Wire 1 50% straight	Upper	.016x.016 CuNITI / NITI	789845990173	02-08-2018	02-22-2018	05-24-2018	Automatic
Skomerza	Mary	S4H2	183103	Wire 1 50% straight	Lower	.016x.016 CuNITI / NITI	789845990173	02-08-2018	02-22-2018	05-24-2018	Automatic
Skomerza	Mary	S4H2	183104	Wire 2 100% straight	Upper	.016x.016 CuNITI / NITI	789845990173	02-08-2018	02-22-2018	05-24-2018	Automatic
Skomerza	Mary	S4H2	183105	Wire 2 100% straight	Lower	.016x.016 CuNITI / NITI	789845990173	02-08-2018	02-22-2018	05-24-2018	Automatic
HORSEY	Tracey	X6N7	192893	WS 2 Plan 2 SS#1 Mx 17*25 NITI Exp 5mm Mb 16*22 NITI	Lower	.016x.016 CuNITI / NITI	781054587373	05-09-2018	05-21-2018	05-24-2018	Renee Phillips
Giffing	Ashley	Q8N6	208028	Wire 1 100% U/L 016N	Upper	.016 round CuNITI / NITI	782980153177	09-17-2018	09-28-2018	12-28-2018	Automatic
Giffing	Ashley	Q8N6	208029	Wire 1 100% U/L 016N	Lower	.016 round CuNITI / NITI	782980153177	09-17-2018	09-28-2018	12-28-2018	Automatic
Giffing	Ashley	Q8N6	208032	Wire 2 100% U/L 16x16N	Upper	.016x.016 CuNITI / NITI	782929502466	09-17-2018	09-25-2018	12-25-2018	Automatic
Giffing	Ashley	Q8N6	208033	Wire 2 100% U/L 16x16N	Lower	.016x.016 CuNITI / NITI	782929502466	09-17-2018	09-25-2018	12-25-2018	Automatic
Berggren	Christina	O3F3	212907	Wire 1 100% U/L 016N	Upper	.016 round CuNITI / NITI	783602618665	10-29-2018	11-05-2018	02-04-2019	Automatic
Berggren	Christina	O3F3	212908	Wire 1 100% U/L 016N	Lower	.016 round CuNITI / NITI	783602618665	10-29-2018	11-05-2018	02-04-2019	Automatic
Berggren	Christina	O3F3	212909	Wire 2 100% U/L 16x16N	Upper	.016x.016 CuNITI / NITI	783602618665	10-29-2018	11-05-2018	02-04-2019	Automatic
Berggren	Christina	O3F3	212910	Wire 2 100% U/L 16x16N	Lower	.016x.016 CuNITI / NITI	783602618665	10-29-2018	11-05-2018	02-04-2019	Automatic
VANDERSLIJ	Marcel	V4H3	5001721702	But update wire 5 19*25 TMA 100%	Upper	.019x.025 Beta-Ti	789619610782	02-01-2018	02-06-2018	02-27-2018	Renee Phillips
VANDERSLIJ	Marcel	V4H3	5001721719	But update wire 5 19*25 TMA 100%	Lower	.019x.025 Beta-Ti	789619610782	02-01-2018	02-06-2018	02-27-2018	Renee Phillips


- The software lists items on the Shipments tracking page as soon as the order is submitted. The most popular functions on the Shipments page are:
- Check on the FedEx® status of a shipment
- Note when items are relocated to a satellite office
- Check that items such as wires are on hand for an upcoming appointment

Filter by Shipment Status

The software lists items on the Shipments tracking page as soon as the order is submitted.

You can view all shipments being tracked or you can filter for:

- **Pending Shipments** - they are ordered but not shipped yet.
- **Shipments in Transit** - they have shipped and can be tracked via a link to FedEx to determine their estimated arrival date.
- **Shipments Received** - they have been checked in by your office and are on hand for the appointment. You can also indicate if you move the wires to a satellite office where the patient is scheduled for the wire insertion appointment.

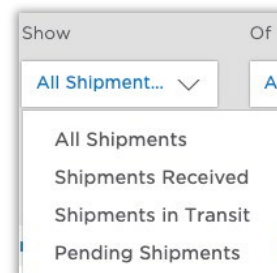


Your wire prescriptions are consolidated into weekly shipments. As packages arrive at your office, check them in using the SureSmile web software.

TIP: If the practice does not check in the wires, they are automatically removed after 90 days from the “Shipments In Transit” list and appear on the “Shipments Received” list. In the Received By column, “Automatic” will be listed instead of a username.


Filter by Shipment Type

You can view all shipments from SureSmile®, or you can filter for type of shipment such as wires.

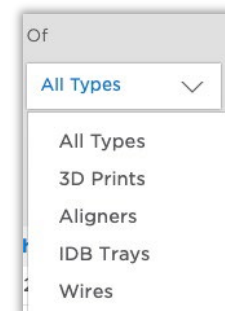

Filter by Treating Doctor

You can view shipments regardless of treating doctor, or you can filter for a specific doctor's patients only.

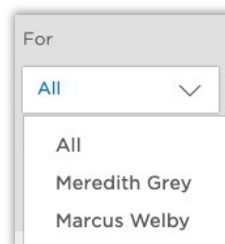

Retire Tracking Information

Tracking information is removed automatically from the Shipments page under these conditions:

- The status of the patient record is changed to *finished*.
- The wire status is updated to *inserted* or *not used*.
- If neither of these state changes occurs, the software has a cleanup function. It removes tracking information from the shipments page automatically after 730 days (two years).

Checking in Wires

Check in Shipments with a Bar Code Scanner

For the most efficient and error-free method of checking in SureSmile shipments, scan the FedEx label using a bar code scanner. Each wire order contained in the box will be updated on the shipments tracking page.



Tip: You do not need to contact SureSmile Customer Care to set up your bar code scanner with SureSmile. Any Windows-compatible equipment will work immediately after it is installed.

1. From the clinic overview, select the **Shipments** tab.
2. In the Show filter, select **Shipments In Transit** to display a list of all of your wire orders that have been shipped (or you can view the **All Shipments** list).
3. In the **Of...Type** filter, select **Wires**.
4. If needed, filter by treating doctor.
5. If you have multiple offices, select the location for the corresponding wire appointments. (Skip this step if the wires in the package may go to various locations.)
6. Click in the box adjacent to the *Receive* button to insert your cursor.

A screenshot of the SureSmileWeb interface. It shows a search bar with the text "Enter tracking # or product ID to receive..." and a blue "Receive" button. The search bar is part of a larger interface element that also includes a dropdown menu showing "SureSmileWeb E6.1".

7. Use a bar code scanner to scan the bar code on the FedEx label. If you received multiple wire prescriptions in a box, scanning the barcode on the outside of the box will capture the information for all of the contents.

TIP: After checking in a box, you can press <F5> to update the list (which removes the orders you have checked in when you are filtered for "Shipments in Transit").

View Received Wires

Notice the Received Date is updated and your username is entered in the Received By column.

1. To see the wires you have received, select **Show Shipments Received** in the filter.
2. In the **Of...Type** filter, select **Wires**.
3. If needed, filter by treating doctor.

Choose a bar code scanner that can conveniently connect to one of your SureSmile computers through a USB port.

You can find bar code scanners at your local electronics store or online at sites such as Amazon.

Here are two brands used by some of our SureSmile practices:

1. Adesso NuScan1000U USB Handheld Contact CCD Barcode Scanner
2. Unitech MS210 - Bar Code Scanner



Check in FedEx Boxes Manually

If you do not have a bar code scanner, you can copy/paste the tracking number into the software to receive the contents. There is no need to type, which runs the risk of introducing typos (errors).

1. From the clinic overview, select the **Shipments** tab.
2. In the Show filter, select **Shipments In Transit** to display a list of all of your orders that have been shipped.
3. In the **Of...Type** filter, select **Wires**.
4. If you have multiple offices, select the location for the corresponding patient appointments. (Skip this step if the wires in the package may go to various locations.)
5. Click the **Tracking Number** column header to sort by most recent tracking number and scroll down to find the number that matches the package label.

6. Drag your mouse across the number to highlight it and then right-click and choose **Copy**.
7. Click in the box adjacent to the Receive button to insert your cursor, and then right-click and choose **Paste**.
8. Click **Receive**.

Tracking Number	Order Date	Shipped Date	Rece
78984599			05-2
78984599			05-2
78984599			05-2
78984599			05-2
781054587			05-2
782980153			12-28

TIP: To see the items you just received, select **Show Shipments Received** in the filter.

Check in Wire Boxes Manually

1. If you are checking in one wire package, you can copy/paste the wire ID. There is no need to type, which runs the risk of introducing typos (errors).
2. From the clinic overview, select the **Shipments** tab.
3. Use the quick search feature to filter for the patient ID.
4. If you have multiple offices, select the location for this patient's wire appointment. (Or you can enter this information later - see page 53.)
5. Find the wire order that matches the prescription package.

TIP: You can click the **Product ID** column header to sort in ascending order, and then click it again to reverse sort.

Product ID	Product Name
183102	Wire 1.50% straight
183103	
183104	
183105	

6. Drag your mouse across the Product ID number to highlight it and then right-click and choose **Copy**.
7. Click in the box adjacent to the *Receive* button to insert your cursor, and then right-click and choose **Paste**.
8. Click **Receive**.

TIP: To see the items you just received, select **Show Shipments Received** in the filter.

Select Destination Site (Multi-Site Practices Only)

If you have multiple offices, your wires are shipped to the office location where you created the patient record. Or, you can contact Customer Care to request that all of your shipments go to one address. If a patient's wire appointment is scheduled in a different office than the one that received the package, you will transport the wires. To help you track moved wires, SureSmile allows you to indicate the site location on the Shipments page.

1. From the clinic overview, select the **Shipments** tab.
2. Enter the patient ID in the quick search or filter as needed to find the wires you are transporting.
3. Under the Site column, use the drop-down menu to select the location for the wire.

NOTE: You cannot select a site until the wire is shipped.

Print Tracking Information

You can print the page you are viewing using your browser's print function.

TIP: For Google Chrome, right-click on the page and choose **Print**.

Last Name	First Name	Patient ID	Product ID	Product Name	Upper/Lower	Cross Section	Tracking Number	Order Date	Shipped Date	Receive Date	Receive By
Skomerza	Mary	S4H2	183102	Wire 1 50% straight	Upper	.016x.016 CuNITI / NITI	789845990173	02-08-2018	02-22-2018	05-24-2018	Automatic
Skomerza	Mary	S4H2	183103	Wire 1 50% straight	Lower	.016x.016 CuNITI / NITI	789845990173	02-08-2018	02-22-2018	05-24-2018	Automatic
Skomerza	Mary	S4H2	183104	Wire 2 100% straight	Upper	.016x.016 CuNITI / NITI	789845990173	02-08-2018	02-22-2018	05-24-2018	Automatic
Skomerza	Mary	S4H2	183105	Wire 2 100% straight	Lower	.016x.016 CuNITI / NITI	789845990173	02-08-2018	02-22-2018	05-24-2018	Automatic
HORSEY	Tracey	X6N7	192893	WS 2 Plan 2 SS#1 Mx 17*25 NITI Exp 5mm Mb 16*22 NITI	Lower	.016x.016 CuNITI / NITI	781054587373	05-09-2018	05-21-2018	05-24-2018	Renee Phillips
Giffing	Ashley	Q8N6	208028	Wire 1 100% U/L 016N	Upper	.016 round CuNITI / NITI	782980153177	09-17-2018	09-28-2018	12-28-2018	Automatic
Giffing	Ashley	Q8N6	208029	Wire 1 100% U/L 016N	Lower	.016 round CuNITI / NITI	782980153177	09-17-2018	09-28-2018	12-28-2018	Automatic
Giffing	Ashley	Q8N6	208032	Wire 2 100% U/L 16x16N	Upper	.016x.016 CuNITI / NITI	782929502466	09-17-2018	09-25-2018	12-25-2018	Automatic
Giffing	Ashley	Q8N6	208033	Wire 2 100% U/L 16x16N	Lower	.016x.016 CuNITI / NITI	782929502466	09-17-2018	09-25-2018	12-25-2018	Automatic
Berggren	Christina	O3F3	212907	Wire 1 100% U/L 016N	Upper	.016 round CuNITI / NITI	783602618665	10-29-2018	11-05-2018	02-04-2019	Automatic
Berggren	Christina	O3F3	212908	Wire 1 100% U/L 016N	Lower	.016 round CuNITI / NITI	783602618665	10-29-2018	11-05-2018	02-04-2019	Automatic
Berggren	Christina	O3F3	212909	Wire 2 100% U/L 16x16N	Upper	.016x.016 CuNITI / NITI	783602618665	10-29-2018	11-05-2018	02-04-2019	Automatic
Berggren	Christina	O3F3	212910	Wire 2 100% U/L 16x16N	Lower	.016x.016 CuNITI / NITI	783602618665	10-29-2018	11-05-2018	02-04-2019	Automatic
VANDERSLIK	Marcel	V4H3	5001721702	But update wire 5 19*25 TMA 100%	Upper	.019x.025 Beta-Ti	789619610782	02-01-2018	02-06-2018	02-27-2018	Renee Phillips
VANDERSLIK	Marcel	V4H3	5001721719	But update wire 5 19*25 TMA 100%	Lower	.019x.025 Beta-Ti	789619610782	02-01-2018	02-06-2018	02-27-2018	Renee Phillips


Finding Wire Shipments

Find Missing Wires

If you are preparing for patient appointments and do not have the wires on hand, you can use the Shipments page to determine:

- Were the patient's wires ordered?
- Have the wires shipped?
- If the wires have shipped, when are they due to arrive?
- Were they moved to a satellite office?

1. From the clinic overview, select the **Shipments** tab.

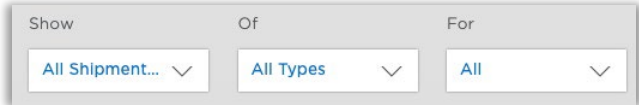

2. In the filters, choose **Show All Shipments, Of Wires** and **For All**.

*TIP: If you know you have assigned a treating doctor to the patient's record (on the Patient Profile page accessible from the Edit button), filter by the doctor's username instead of **For All**.*

OR

Use the quick search feature to filter for a patient ID, if looking for a specific patient.

3. Refer to the table to find your wires:

Check here..	...and then do this:
If there are no wires listed for this patient, most likely they were not ordered.	To check if wires were ordered, go to the Patient's record and view the full timeline. <i>TIP: Wires are removed from the Shipments page when the patient record status is changed to finished or the wire timeline status is changed to inserted or not used.</i>
Check the Received By column and Site (if applicable). If the wire was received, the user who was logged on is listed in the Received By column.	If the wire was received, it is in your office, or the location indicated in the Site column.

Check the Shipped Date column. If this area is blank, the wires have not shipped yet.	Contact Customer Care for your options.
If the Shipped Date column lists a date, the wires shipped.	Click the tracking number link. The FedEx web page opens in a new tab.

TIP: You can click the column headers to sort the list on the columns. For example:

- Click *Patient ID* to sort the list alphabetically by patient ID, starting with AA. Click the column header again to reverse the order to start with Z names.
- Click *Shipped Date* to sort the list by the date the wires were sent to you, starting with the oldest wires. Click the column header again to reverse sorting and list the most recent dates first.

If you have multiple pages of wire orders, click the page numbers to move through the entire list.

Check FedEx Status

At the top of the FedEx web page, you can see if the package was delivered and who signed for it. Otherwise, this area will indicate the package is in transit. The time of delivery (actual or estimated) is indicated in the upper-right corner.

The screenshot shows the FedEx Tracking page for tracking number 770053515186. The status is 'Delivered' with a signature for M. TORRES. The shipment was shipped from Richardson, TX, US on 5/21/2014 at 4:47 pm and delivered to Plainfield, IL, US on 5/23/2014 at 2:37 pm. The travel history table shows the following activities:

Date/Time	Activity	Location
- 5/23/2014 - Friday		
2:37 pm	Delivered	Plainfield, IL
7:17 am	On FedEx vehicle for delivery	AURORA, IL
7:13 am	At local FedEx facility	AURORA, IL
- 5/22/2014 - Thursday		
6:06 pm	At destination sort facility	CHICAGO, IL
4:24 pm	Departed FedEx location	MEMPHIS, TN
7:42 am	Arrived at FedEx location	MEMPHIS, TN
- 5/21/2014 - Wednesday		
9:24 pm	Left FedEx origin facility	GARLAND, TX
4:47 pm	Picked up	GARLAND, TX
1:24 pm	Shipment information sent to FedEx	

The Shipment Facts section includes:

Tracking number	770053515186	Service	FedEx 2Day
Weight	3 lbs	Dimensions	18x12x4 in.
Delivered To	ReceptionistFront Desk	Total pieces	1
Total shipment weight	3 lbs / 1.36 kgs	Packaging	FedEx Envelope
Special handling section	Deliver Weekday		

Trouble-shooting Wire Tracking

I accidentally clicked the Receive button for the wrong wire order. Can I undo it?

No. For the best speed and accuracy, we recommend purchasing a bar code scanner.


Does checking wires in as “received” update reminders in the Patient Overview?


No, the purpose of the “Insert Wire” task is to update the timeline date for the wire insertion milestone. When you see this task, it is a good idea to update each patient’s Wire Insertion date from Estimated to Scheduled who has a definite appointment date. To ensure that the wires are in for approaching appointments, you can run a report of that week’s appointments from your PMS and check off patients whose wires are marked as received in SureSmile®.


Learning Resources

Search the **Help Center** in SureSmile for:

- Online help topics with step-by-step instructions
- Downloads including forms and reference documents
- Video demonstrations of software features

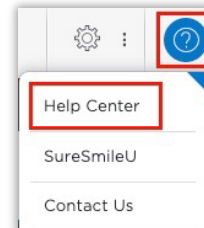



To open the Help Center, click the question mark  button and select **Help Center**.

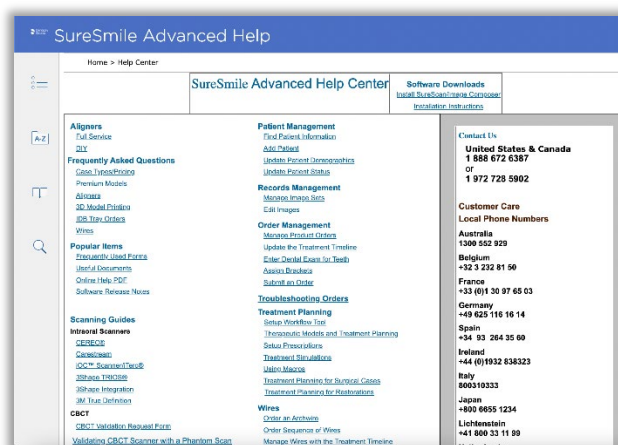

Visit www.suresmileu.com for online learning resources including:

- Archived SureSmile webinars
- Recordings of presentations from events
- Class materials

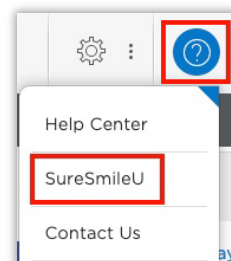

To open SureSmileU from the SureSmile software, click the question mark  button and select **SureSmileU**.

