



New Enhancements in SureSmile January Release Aspen Retainers

SureSmile – Dentsply Sirona 2021

SureSmile® Retainers

- SureSmile Retainers are ordered as an extension of the aligner treatment to keep teeth fixed at their final designed positions
- Retainers are designed based on the final “R” stage of the treatment after attachments have been removed from the patient’s teeth
- SureSmile Retainers are made of a thicker material than aligners: Essix Plus .040



Aspen Retainers January Release


An “Order Retainers” task will appear on your task list 6 weeks before the end of treatment if retainers have not been previously ordered.

The screenshot shows the 'Patient Database Tasks' interface for 'SureSmile Aligner'. At the top, there is a search bar for 'Patient ID' and a 'New Patient' button. Below this is a navigation bar with tabs for 'Tasks (1)', 'Patients', 'Shipments', 'Jobs', 'DS CEREC Orders', and 'Reports'. The 'Tasks (1)' tab is selected. To the right, there are filters for 'Sort by' (Task), 'Show Tasks of' (Reminder), 'Tasks for' (empty), and 'Show' (Active Tasks). The main area displays a table of tasks with the following columns: Flag, Card, Last name, First name, Patient ID, Owner, Item, Task, Due Date, Notes, Setup Approval Date, Status, and Deferred To. A single task is listed: 'Order Retainers' for patient 'G4M3', with a due date of '2021-12-22' and a status of 'Active'. The 'Order Retainers' text in the task list is highlighted with a red rectangular box.

Flag	Card	Last name	First name	Patient ID	Owner	Item	Task	Due Date	Notes	Setup Approval Date	Status	Deferred To
<input type="checkbox"/>		Patient	Aligner	G4M3			Order Retainers <input type="checkbox"/>	2021-12-22		2021-12-08	Active	


Aspen Retainers Prior to January Release

Prior to the January release, the “Include retainers with aligners” check box is available during initial aligner order

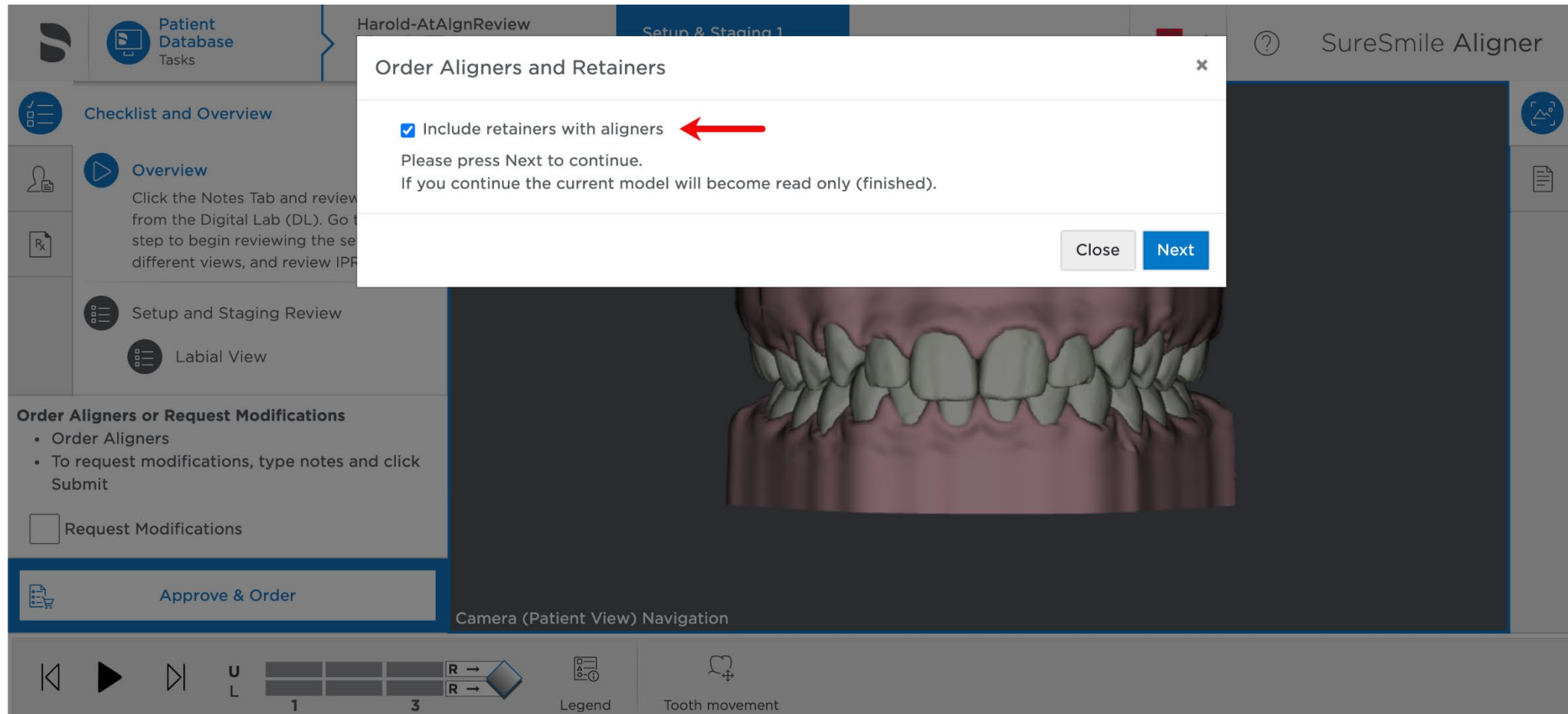

Aspen Retainers January Release

“Include retainers with aligners” check box is no longer available during initial aligner order. Retainers will be available to order after the initial aligners have been ordered.

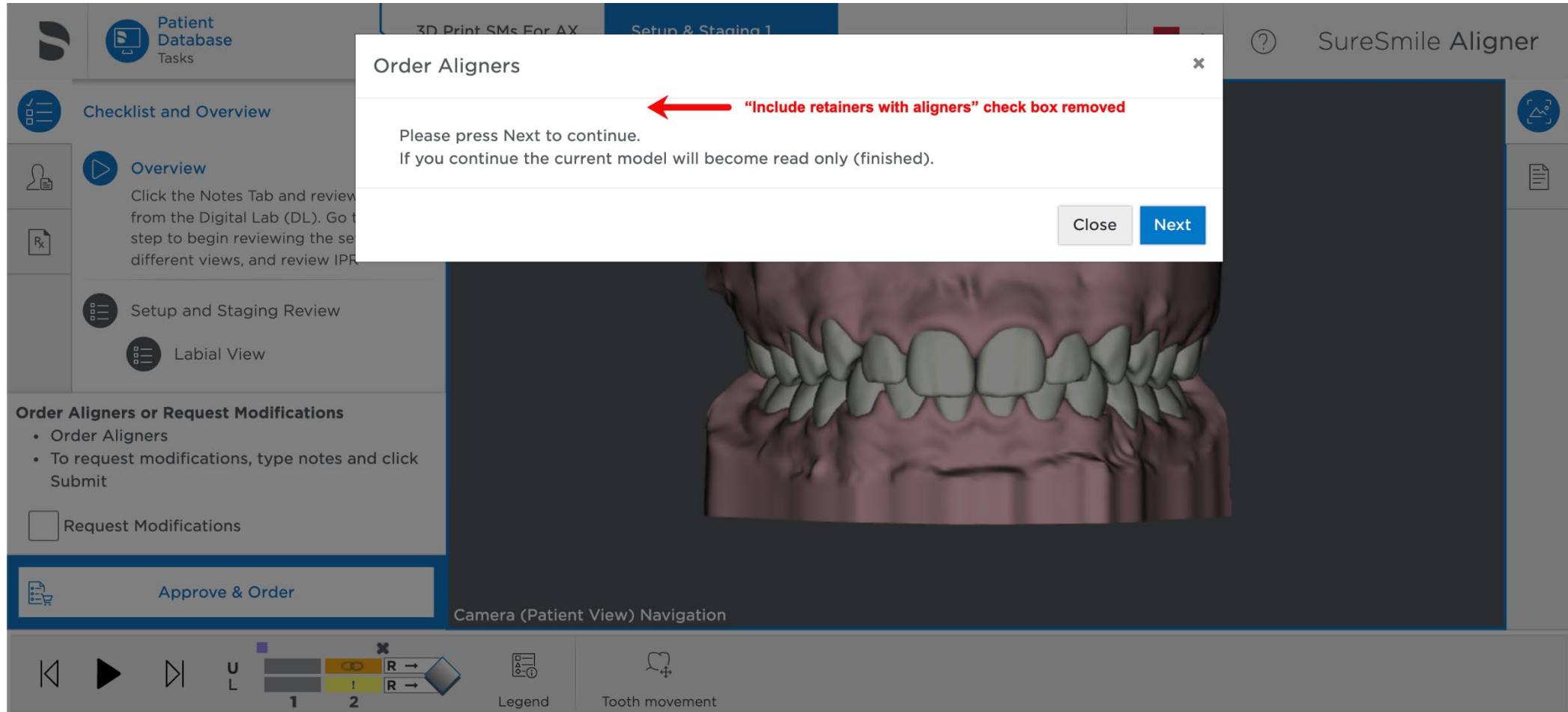

Aspen Retainers Prior to January Release

Prior to the January release, the “Order Aligners” button is used to order retainers after initial aligner order.

The screenshot displays the SureSmile Aligner software interface. At the top, there is a navigation bar with a 'Patient Database Tasks' icon, a search field, and a 'Setup & Staging 1 (Approved / Approved)' dropdown menu. The main interface is divided into several panels. On the left, a 'Checklist and Overview' panel lists various views: Overview (checked), Setup and Staging Review (checked), Labial View (checked), Right Buccal View (checked), Left Buccal View (checked), Upper Occlusal (checked), and Lower Occlusal (checked). Below this list, the 'Order Aligners' button is highlighted with a red border. Below that, the 'Order Refinement (\$)' button is highlighted with a blue border. The central area shows a 3D model of a patient's teeth with yellow outlines, and a 'Camera (Patient View) Navigation' control below it. On the right, there is an 'Image Assets' panel showing a blurred photo of a patient's face and a clear photo of the patient's teeth. At the bottom, there is a control bar with navigation icons (back, play, forward), a legend, a 'Tooth movement' section with 'U' and 'L' labels, and a 'Get report' button. A red arrow points from a text box to the 'Order Aligners' button.

“Order Aligners” button is used to order retainers after Setup & Staging is approved.


Aspen Retainers January Release

New “Order Retainers” button is used to order retainers after initial aligner order.

The screenshot displays the SureSmile Aligner software interface. At the top, the navigation bar includes 'Patient Database Tasks', 'Setup & Staging 1 (Approved / Approved)', and 'SureSmile Aligner'. On the left, a 'Checklist and Overview' panel lists several completed tasks: Overview, Setup and Staging Review, Labial View, Right Buccal View, Left Buccal View, and Upper Occlusal. The 'Order Retainers' button is highlighted with a red border and a red arrow points to it. A text box below the arrow states: 'New “Order Retainers” button appears after Setup & Staging is approved.' The central area shows a 3D dental model with a yellow retainer wire and a blue square on the upper teeth. To the right, there is an 'Image Assets' panel with a patient photo and a close-up of the teeth. At the bottom, there is a control bar with navigation icons, a legend, and a 'Get report' button.


Aspen Retainers Prior to January Release

Prior to the January Release, the “Order Aligners and Retainers” dialog allows you to order one or both retainers.

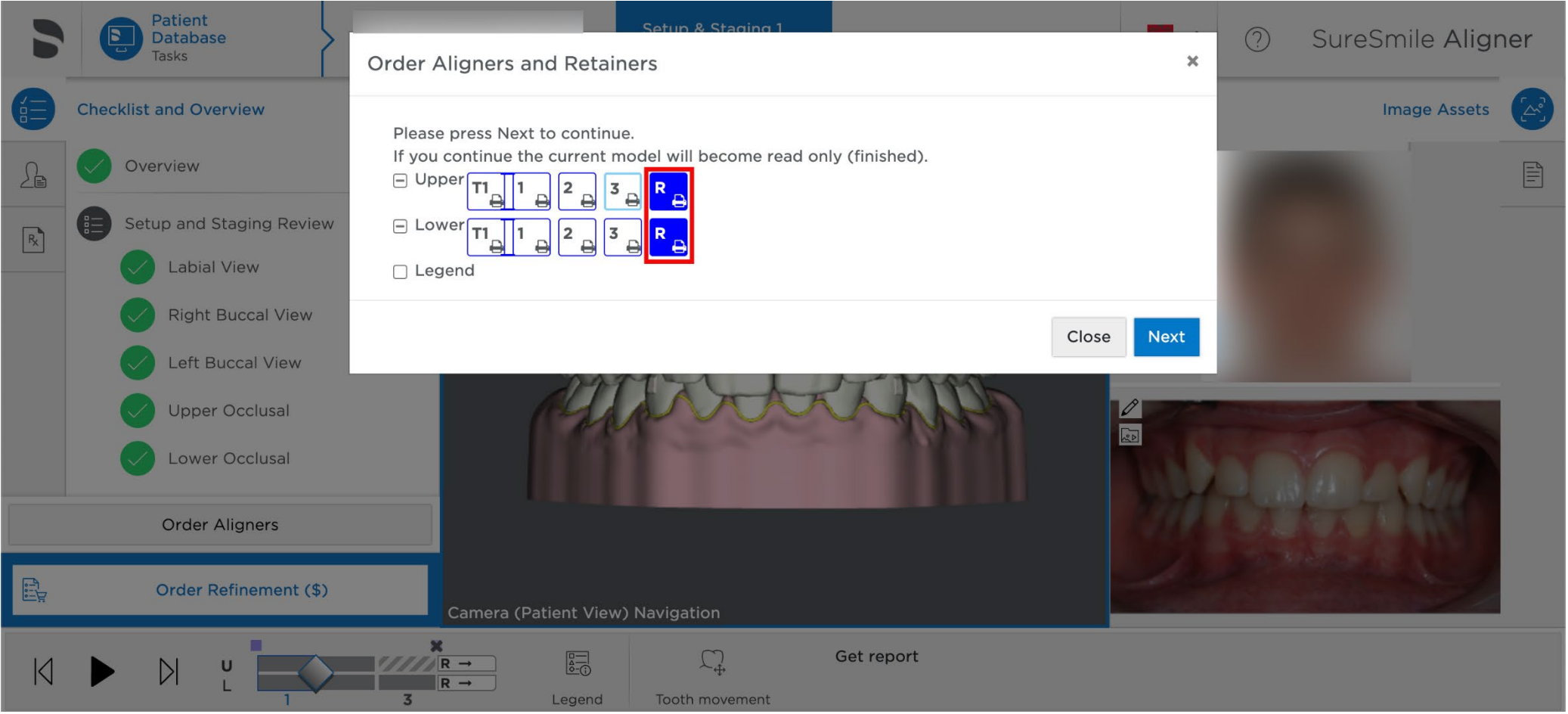

Aspen Retainers January Release

New “Order Retainers” pop up allows you to order retainers for one or both arches from the R stage in quantities of 1-5.

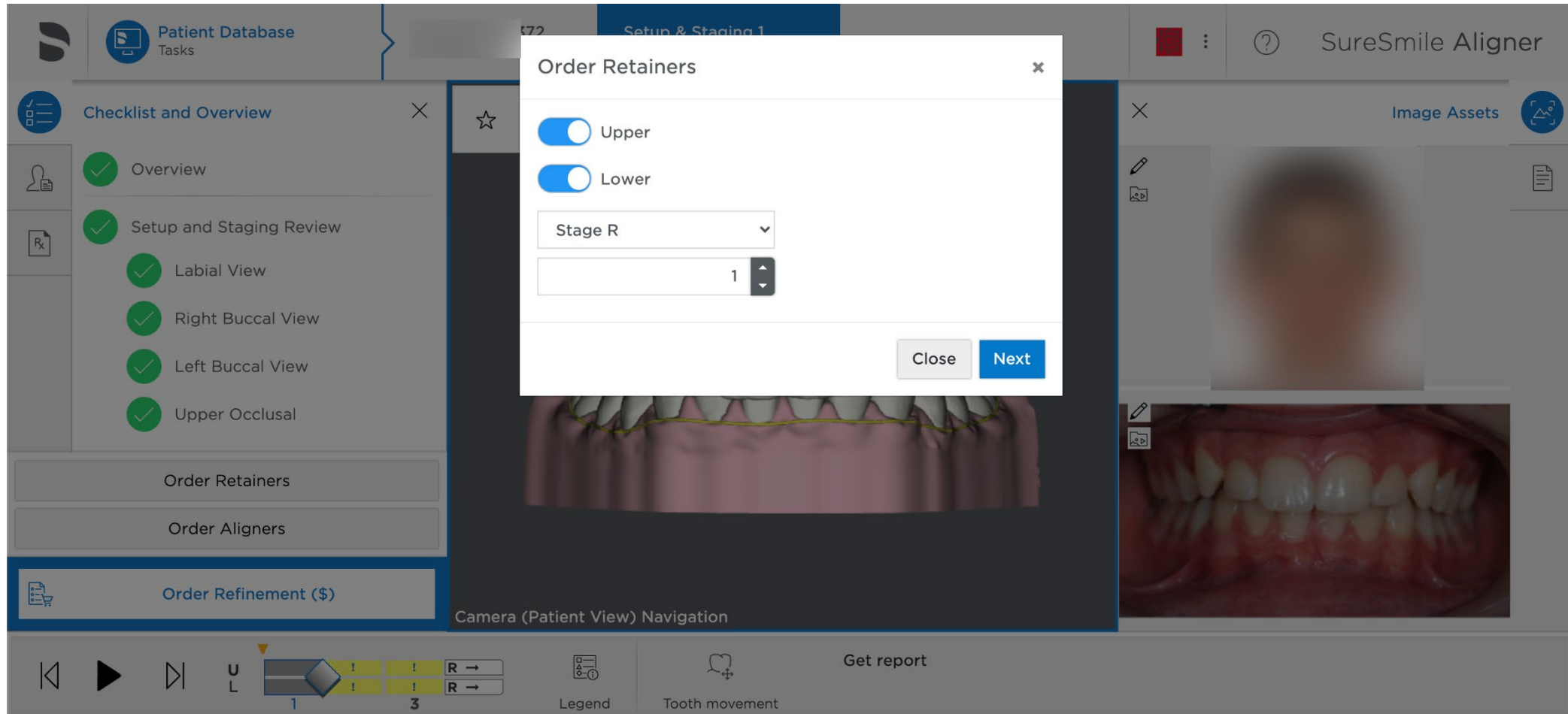

Aspen Retainers January Release

Once retainers have been ordered, they will show in the patient timeline similar to how aligners show today.

The screenshot displays a patient's timeline with various events. The 'Retainers for [Setup & S... Ordered: 2021-12-13' entry is highlighted with a red box. The interface includes a patient profile section on the left and a detailed timeline on the right.

Event	Date
Treatment Start	2021-12-13
Therapeutic Model 1	Approved: 2021-12-13
Setup & Staging 1	Approved / Approved: 2021-12-13
Aligners for [Setup & St...	Ordered: 2021-12-13
Retainers for [Setup & S...	Ordered: 2021-12-13
Setup Approval Date	2021-12-13
Treatment Completion	2022-03-04

Thank You



All rights reserved. SureSmile is a registered trademark of Dentsply Sirona, Inc.
© 2021 Dentsply Sirona, Inc.

0281-EDUMAT-500577-A Rev 18
Issued 01-2022
