

# New Enhancements in SureSmile January Release Retainers

SureSmile – Dentsply Sirona 2022

# SureSmile® Retainers

- SureSmile Retainers are ordered as an extension of the aligner treatment to keep teeth fixed at their final designed positions
- Retainers are designed based on the final “R” stage of the treatment after attachments have been removed from the patient’s teeth
- SureSmile Retainers are made of a thicker material than aligners: Essix Plus .040
- There will be an additional fee (per arch) for each SureSmile Retainer



# SureSmile® Retainers


An “Order Retainers” task will appear on your task list 6 weeks before the end of treatment if retainers have not been previously ordered.

The screenshot shows the SureSmile Ortho software interface. At the top, there is a navigation bar with the SureSmile logo, 'Patient Database Tasks', a search bar, a filter icon with '1', a 'New Patient' button, a settings gear, and the text 'SureSmile Ortho'. Below the navigation bar is a filter section with tabs for 'Tasks (2)', 'Patients', 'Shipments', 'Jobs', and 'Reports'. To the right of these tabs are dropdown menus for 'Sort by' (set to 'Task'), 'Show Tasks of' (set to 'Reminder'), 'Tasks for' (set to 'All Staff'), 'Show' (set to 'Active Tasks'), and 'View' (set to a list view). Below the filter section is a table with the following columns: Flag, Card, Last name, First name, Patient ID, Owner, Item, Task, Due Date, Notes, Setup Approval Date, Status, and Deferred To. The table contains one row with the following data: Flag (checkbox), Card (eye icon), Last name (Test-077-Alig...), First name (redacted), Patient ID (V5S7), Owner (Staff Test), Item (redacted), Task (Order Retainers with a plus icon and a checkbox), Due Date (2021-12-08), Notes (document icon), Setup Approval Date (-----), Status (Active), and Deferred To (redacted). The 'Order Retainers' task is highlighted with a red box.

Flag	Card	Last name	First name	Patient ID	Owner	Item	Task	Due Date	Notes	Setup Approval Date	Status	Deferred To
<input type="checkbox"/>		Test-077-Alig...		V5S7	Staff Test		Order Retainers <input type="checkbox"/>	2021-12-08		-----	Active	

# SureSmile® Retainers


Retainers can be ordered after the initial aligner order has been placed. The aligner for the R stage will no longer be available for order. Once the Setup & Staging / Staged Models has been approved, you will see an “Order Retainers” button.

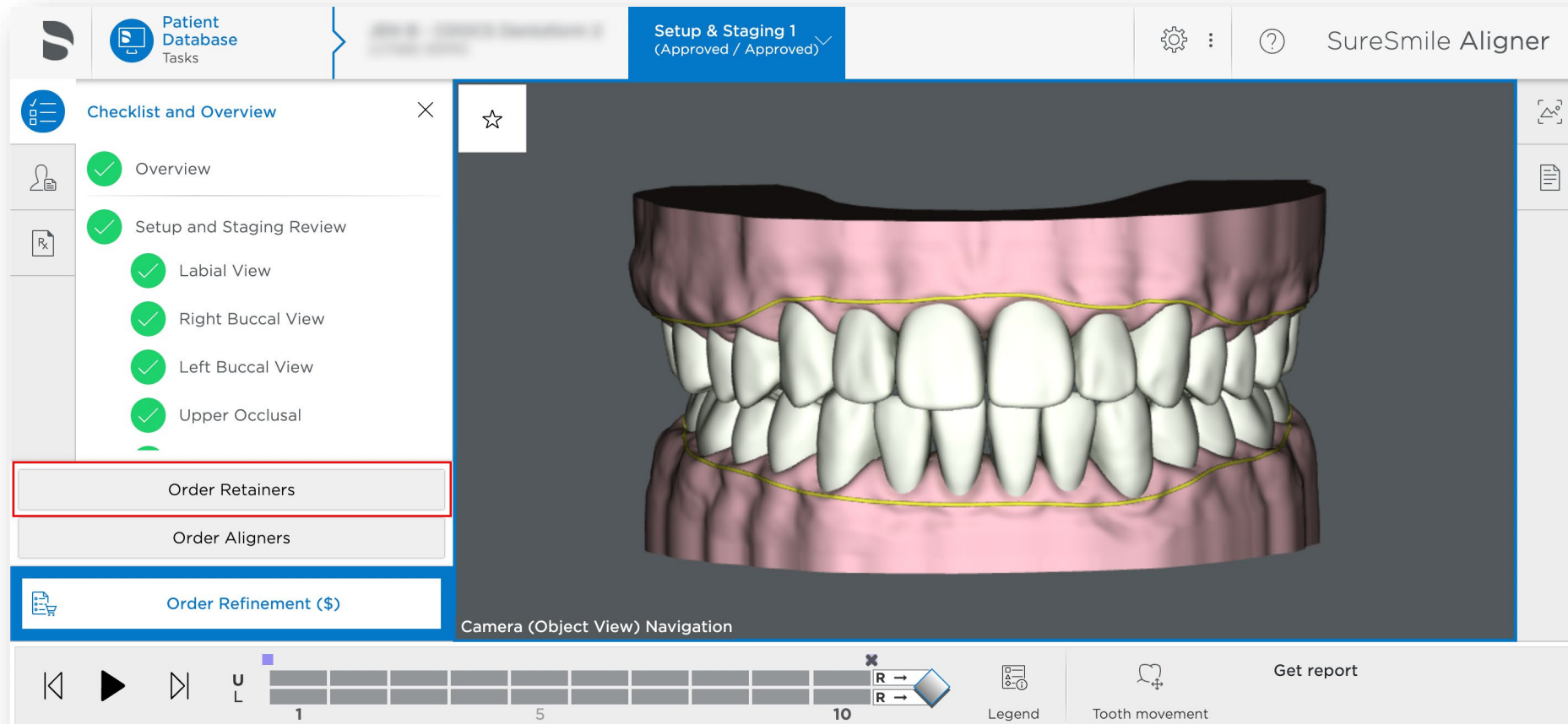




# SureSmile® Retainers from Refinements

For refinements, you have the option to Approve & Order Retainers before ordering aligners.

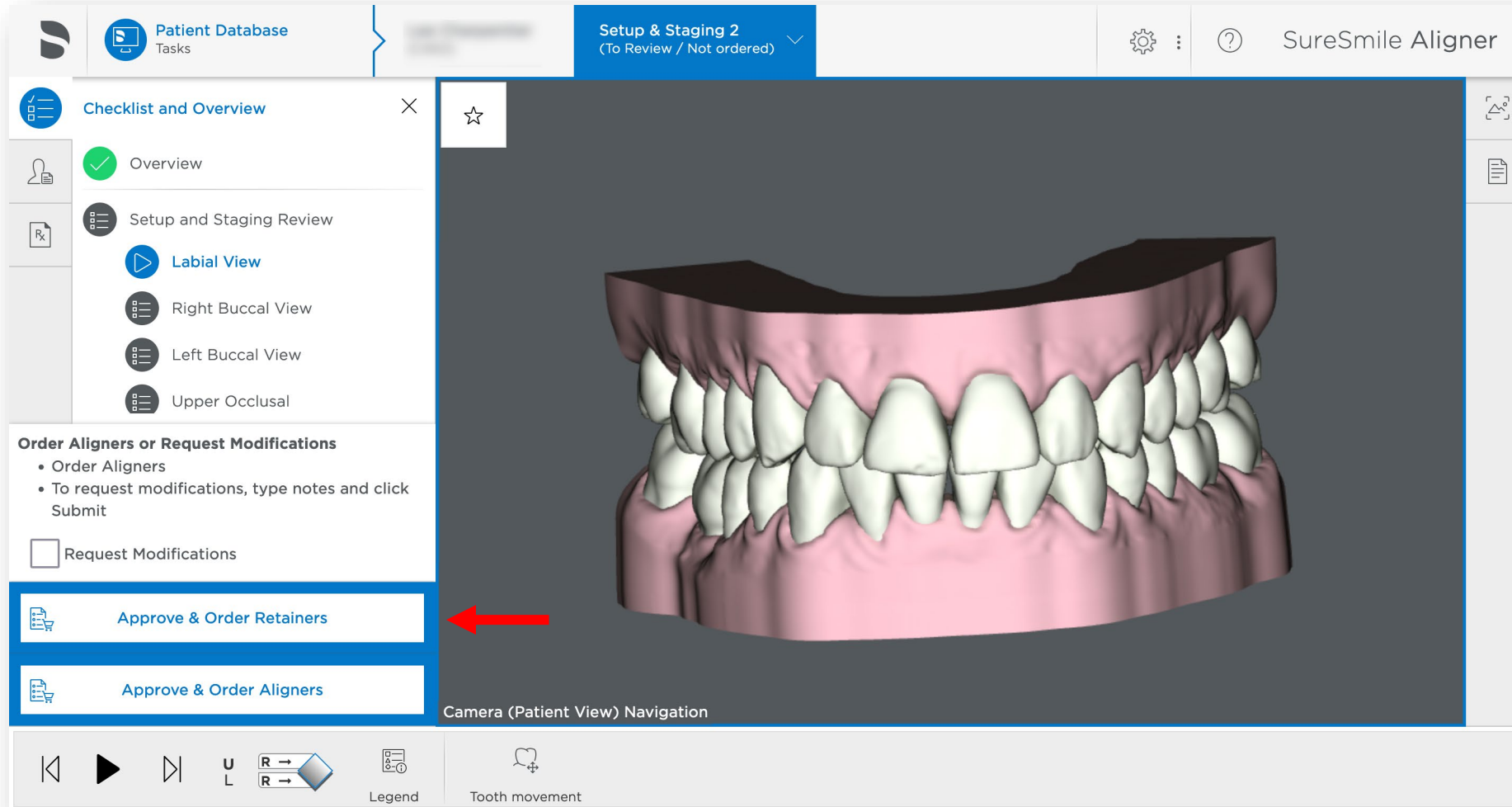



# SureSmile® Retainers

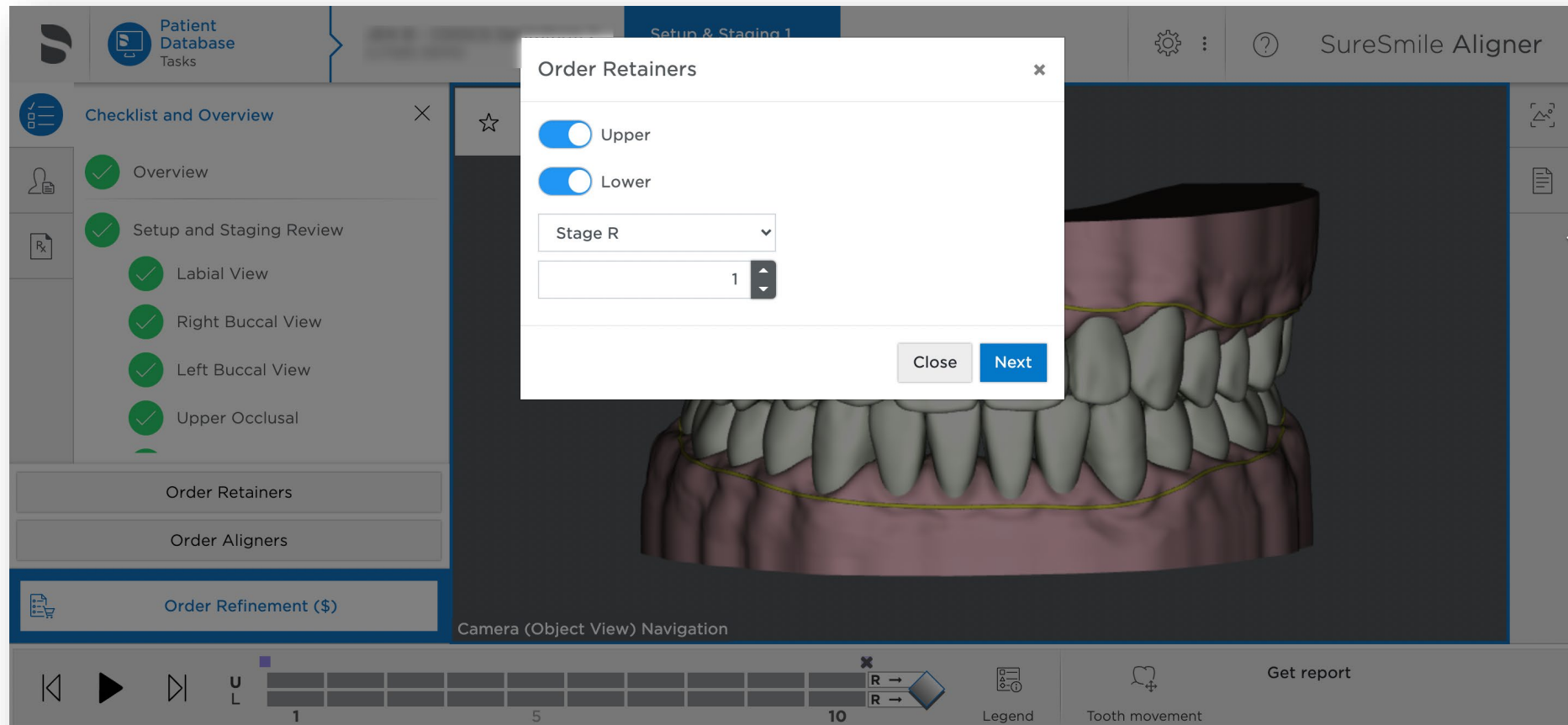Retainers can be ordered for one or both arches from the R stage in quantities of 1-5.



# SureSmile® Retainers

Once retainers have been ordered, they will show in the patient timeline similar to how aligners show today.



Staged Models 1 2021-12-06  
Approved

ID: O8V8

### Recent Notes

Edit Note

Patient Timeline ×

✓	Diagnostic Model 1	Approved: 2021-12-04
!	Simulation 1	2021-12-04
✓	Staged Models 1	Approved: 2021-12-06 ▾
✓	Aligners for [Staged Mod...]	Ordered: 2021-12-06 <
✓	VPro Device for [Staged ...]	Ordered: 2021-12-06
✓	Retainers for [Staged Mo...]	Ordered: 2021-12-06 <

# SureSmile® Retainers

- The SureSmile team will begin manufacturing retainers once they are ordered; they are not held until the end of treatment.
- Processing times for retainers are the same as processing times for aligners. Please see chart to the right for a breakdown by location.
- Standalone retainers (retainers from a new scan and not part of a treatment plan) are not supported at this time but will be coming in a future release.

Destination	Estimated # of days from Order to Delivery
USA	10
Canada	11
Europe	13
Australia	15
New Zealand	16
Japan	13



Thank You



All rights reserved. SureSmile is a registered trademark of Dentsply Sirona, Inc.  
© 2022 Dentsply Sirona, Inc.

0281-EDUMAT-500577-R Rev 20  
Issued 3-2022