



New Enhancements in SureSmile January Release

SureSmile – Dentsply Sirona 2021

User Enhancements in SureSmile January Release

- The January Release includes the following enhancements added since the previous October Release.



- Important Note: After every SureSmile Advanced, Ortho, and Aligner software update, we recommend clearing your browser cache to remove any remnants of the old version to optimize software performance. The steps to follow depend on the browser you are using, and are linked here for your convenience:
 - [Steps for Mozilla Firefox](#)
 - [Steps for Apple Safari](#)
 - [Steps for Google Chrome](#)

User Enhancements in SureSmile January Release

- Interface Improvements (all platforms)
 - Sequential Distalization
 - Enhance the maximum number of stages from 50 to 70
 - Change default trim height for aligners to 0 mm
 - Aligner Templates should not have slits
 - Shipment Email Notification Preference for new registrants
- Russia (Aligner & Ortho)
 - Country Support & Languages

Sequential Distalization for Full-service Aligner Cases

You may now request sequential distalization in your prescription form for the Digital Lab.

**Example note to
Digital Lab**

Aligner Notes:

Please correct the AP relation close to Class I canines and molars as possible by distalizing the bicuspids and molars and canines sequentially to class I and to get resultant less over jet 2-3 mm max

Thank you

Sequential Distalization for SureSmile Advanced DIY

SureSmile Advanced users will have access to the sequential distalization features in DIY cases

Constraints
Select or edit constraints

Aligners Attachments Trim Type

Clinic Single Group Crowding Edit

▼ Accomodation Stage
 Skip accomodation stage

▼ Sequential Distalization

Tooth Range for Distalization:

Teeth 8 .. 4
 Teeth 8 .. 3

Esthetic option: Anterior movement from beginning

No overlapping movement
 Double tooth overlapping movement after 50 %
 Multi tooth overlapping movement after 4 Stages


▼ Groups

Group 1 Rotation Facial Crown Torque Lingual Crown Torque Buccal Movement
Mesial Angulation Distal Angulation Mesial Movement Distal Movement

OK Cancel

Maximum number of stages increased to 70

Prior to this release, if a treatment plan required more than 50 stages, the first 50 stages would be created with the remaining movements in the last stage, and a refinement would be required later to complete the treatment. The software now allows 70 stages instead of 50 stages.

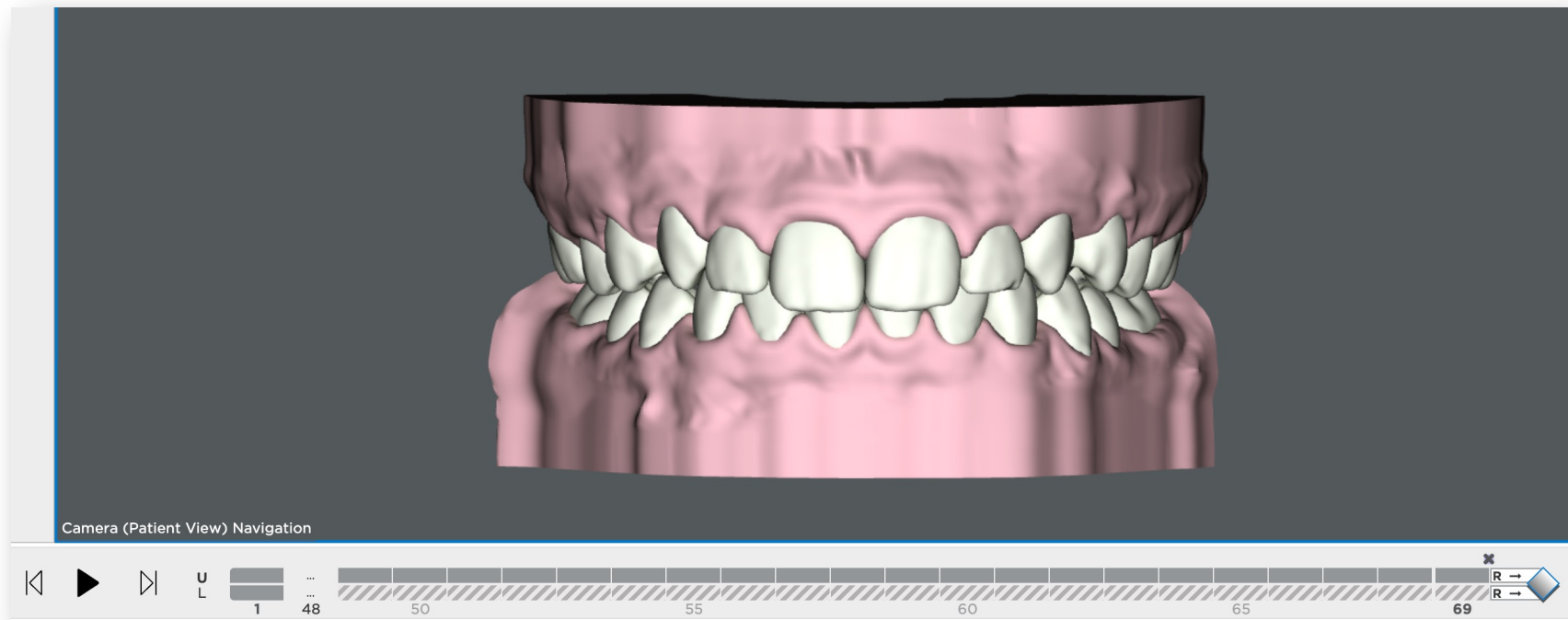

Default trim height for aligners set to 0mm

The default trim height for the straight trim type is changing from 1.0 mm to 0.0 mm for new users. Existing users are not affected.

The screenshot shows the 'Aligners' tab in the SureSmile software. A callout box points to the 'Aligners' tab with the text: "In SureSmile Aligner, Trim Type & Height can be found under the Aligners tab". Below this, the 'Trim Type & Height' section is visible. It contains two rows of settings for the upper (U) and lower (L) arches. Each row has an 'Aligner Trim Type' section with radio buttons for 'Straight' (selected) and 'Scalloped', and a 'Straight Trim Height' section with a dropdown menu set to '0.0 mm'. A 'Save' button is located at the bottom left of the settings area.

Default trim height for aligners set to 0mm

The default trim height for the straight trim type is changing from 1.0 mm to 0.0 mm for new users. Existing users are not affected.

The screenshot shows the 'Trim Type & Height' settings in the SureSmile software. The interface includes a navigation bar with tabs: Others, Bracket Sets, Bracket Height Sets, Aligner Constraints, Trim Type & Height, and another partially visible tab. The main content area is titled 'Trim Type & Height' and contains two sections for 'U' (Upper) and 'L' (Lower) aligners. Each section has an 'Aligner Trim Type' with radio buttons for 'Straight' (selected) and 'Scalloped', and a 'Straight Trim Height' field set to '0.0 mm'. A blue callout bubble points to the 'Trim Type & Height' tab with the text: 'In SureSmile Ortho & Advanced, Trim Type & Height can be found under the Trim Type & Height tab'. A 'Save' button is located at the bottom left of the settings panel.

Aligner templates set to not have slits

Prior to this release, aligner templates had cut-outs. Since cut-outs are not needed in templates, this is being removed.



Shipment Email Notification will default to Doctor for new registrants

The screenshot shows the 'Task Owner' configuration page in the SureSmile Aligner software. The interface includes a top navigation bar with 'Patient II' and a filter icon, and a main menu with 'Prescription', 'Treatment', 'Aligners', 'Task Owner', 'Appearance', 'Flag Labels', and 'Search'. The 'Task Owner' section is active, displaying settings for 'Products', 'Tasks', 'Order Assignments', 'Review/Approve Assignments', and 'Tasks Assignments'. The 'Shipment Email Notification' dropdown menu is open, showing options: 'None', 'Doctor', 'Staff', and 'Both'. The 'Doctor' option is highlighted. A blue callout box on the right contains the text: 'Shipment Email Notification default changing from 'None' to 'Doctor' for new practices moving forward. This change will not affect existing practices at this time.'

Change default for Shipment Email Notification Preference

From: <sureweb@suresmile.com>
Date: Fri, Aug 20, 2021 at 2:40 PM
Subject: ALERT: SureSmile shipment -- Aligners for [Staged Models 1] 1 for H9D6
To:

Add sureweb@suresmile.com to your address book to prevent emails from going to junk or spam

New shipment

You have a new shipment from SureSmile. Navigate to it by clicking the following links.

Shipper tracking: [\[Redacted\]](#)

Shipments tracking: [Aligners for \[Staged Models 1\] 1 for H9D6](#)

Sincerely,
SureSmile Customer Care

Example of the Shipment Email Notification that will be sent.

Russia Supported

The screenshot shows the 'Sign up for an account' page on the SureSmile website. The form includes fields for Full name, Email address, Password, Password confirmation, Country (set to Russian Federation), Timezone (set to GMT-06:00 Central Time (US & Canada)), and Language (set to English (United States)). A 'Sign up' button is at the bottom. Two callouts provide additional information: one on the left states that Russian translations are supported and Russian is available in the language drop-down; one on the right states that sign-up is disabled for the Russian Federation and registration is by invitation only. Two yellow warning boxes are also present: one notes that the service is not available in the user's country, and another notes that self-registration is not available in the selected country, providing a link to the customer support service.

Russian translations are also now supported. Russian is now available under the Language drop-down.

If user chooses Russian Federation from Country list, sign up is disabled. Russian Terms & Conditions and registration is by invitation only.

Thank You



All rights reserved. SureSmile is a registered trademark of Dentsply Sirona, Inc.
© 2021 Dentsply Sirona, Inc.

0281-EDUMAT-500577 Rev 18
Issued 12-2021